- GB Pedal trainer**
- D Pedaltrainer**
- DK Pedaltræner**
- I Pedaliera**

*Item no.*  
*Art. Nr.*  
*Varenr.*  
*Codice*

**272002 & 272003**

## User manual

**CE**

**MOBILEX A/S**  
Grønlandsvej 5  
DK-8660 Skanderborg  
www.mobilex.dk

**GB Pedal trainer - item no. 272002**

**Product description**

Foldable pedal trainer with a 2-colored steel frame. The pedals have safety straps and the pedal resistance can be adjusted.

**Safety requirements**

- Make sure that this user manual is read by all persons using the device. Manufacturer don't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product label.
- Only use the product for the described purpose and only on even and flat surfaces.
- Do not stand on the pedal trainer.
- Use only sitting in a chair with the pedal trainer placed safely on the floor or by training of the arms, only placed safely on a surface of the right height.

**Assembly**

- Remove the device from the box and insert the adjustment knob.
- Turn the legs outwards until they both click in place.

**Adjusting the pedalling resistance**

The adjustment knob is for adjusting the resistance in the pedals. Turn the knob clockwise. Tighten or loosen as desired.

**WARNING!**

Device may get hot by extended use!

**Cleaning & Maintenance**

Normal dirt on metal and plastic parts can be removed with standard cleaning agents. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (no solvents or abrasives).

**Recommended service life:** Normally 10 years depending on use.

**Reuse**

The product is suitable for recycling.

**Disposal of the product**

The product may not be disposed of with household waste but must be brought to the local recycling site.

**GB Pedal trainer - item no. 272003**

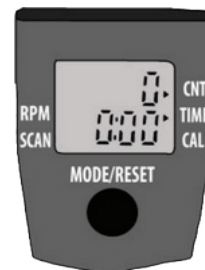
**Model with LCD Display**

The 272003 model differs from the 272002 model on the additional multifunctional LCD display.

**The display provides the following functions:**

- 1. Mode** (red button). This control allows you to select a particular function. To start, press and hold the Red Button for 4 seconds. The LCD will display the different functions (after a few minutes without movement, the unit will automatically shut off).
- 2.** To select a function, wait until the 4 second setup is complete. Push the red button and the arrow will move through the functions. When the desired function is selected, simply use the pedal trainer and the unit will calculate

the exercise.

- 3. Time:** Determines the amount of the time the pedal trainer has been operating.
- 4. Counter (CNT):** Determines the amount of steps while operating the pedal trainer.
- 5. Calorie (CAL):** Determines the amount of calories burned while using the pedal trainer.
- 6. Reps per minute (RPM):** Automatically calculates the repetitions per minute.
- 7. Scan:** Will automatically show the results of all functions.

**Battery replacement**

If the display does not turn on or reads incorrectly, the battery requires replacement.

To replace the battery, use a small screw driver and remove the display panel. Turn the display panel over and remove the watch-style battery. Insert a new battery and insert the display panel again.

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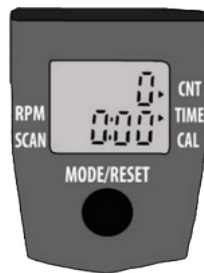
### Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

**Declaration of Conformity (CE)**

(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Danmark, declares that the

**Mobilex pedal trainer**

**Item no. 272002 & 272003**

(Artikel Nr., Varenr., Item no., Codice, varenr.)

**is designed and produced in accordance with the technical documentation and the basic requirements of the:**

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**In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /**

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