- Ⓜ **Support belt with handles - User manual**
- Ⓧ **Stützgurt mit Griffen - Gebrauchsanleitung**
- Ⓡ **Cintura da trasferimento - Manuale d'uso**
- Ⓝ **Støttebælte med håndtag - Brugervejledning**

Item no. - Art. Nr. - Codice - Varenr.

278046 & 278047

CE

MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
Denmark
www.mobilex.dk

General information

This user manual contains a description of the product and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Purpose of the product

The Mobilex support belts with handles can be used in any situation where the carer needs a firm grip in order to assist/support the client. Such situations may include helping a client to move from sitting to standing, and vice versa, and manual handling standing and seated to/from bed, wheelchair, vehicle, commode, toilet, etc. Finally the Support belt can be used to support the client during walking training. The Support belt has a broad contact surface and ergonomical handles, which provide a secure grip from all angles. Easy to use from all sides. Easy to buckle and tighten.

By questions please take contact to your dealer or directly to Mobilex A/S

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 E-Mail: info@mobilex.dk
--	---

Safety requirements

- Make sure that this user manual is read by all persons using the device. Manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- Follow the information on the product label.
- Only use the product for the described purpose.
- Avoid making changes on the product, unless you have the manufacturers written acceptance for such modifications.
- Always check that the label is readable and that the product is in good condition without signs of tear and wear such as loose seams, frays and material inconsistency.
- Do not leave the product unattended on the floor, a person may accidentally step on it with the risk of injury.
- **Warning:** Do not use the product if defects, errors or deterioration is detected. Contact your dealer immediately.

Before and after every use

1. Always perform a risk assessment before starting the specific session with the client in order to make sure the product model is correct and that the procedure is fully understood.
2. Before using the product, read this user manual and the user manual for any additional equipment used.
3. When soiled or stained the product must be replaced or washed according to the product label.
4. After use check the product according to step 1 to 3 above and store in a dry area.

Product care, service and maintenance

The product shall regularly be checked by authorized personnel and always after washing.

Applying the support belt with handles *(please also see the drawings on page 6)*

1. Fit the support belt around the client's waist or hips. Click the buckle closed and adjust the belt so it fits the client.
2. The carer should make sure that the support belt is secure but that it does not irritate the client. The support belt should be retightened during usage.
3. The carer now has a firm grip on the client and can assist/support the client's movements (see Illustration at the end of the manual).

Expected service, storing and shelf life

The product should be stored in a dry and clean environment and the shelf lifetime is 3 years in original packaging. Store in temperatures between 0° and 40° C. Expected service life is 2 years; however, this will be subject to the actual conditions during use.

Technical data

Item no.	Description	Material	Dimensions	Max. user weight
278046	Support belt with handles, Medium	Nylon (PA)	117 x 14 cm	175 kg
278047	Support belt with handles, Large	Nylon (PA)	140 x 14 cm	175 kg

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

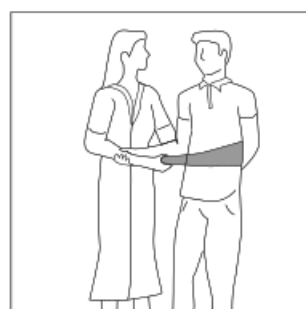
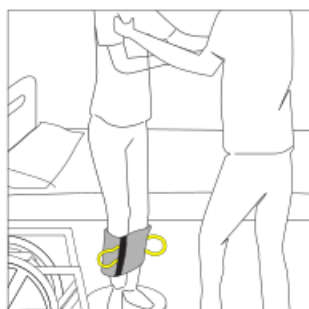
Product label

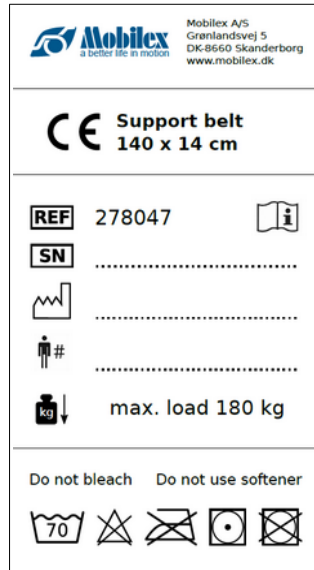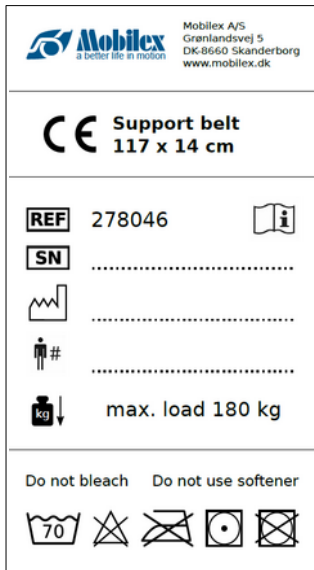The product label is sewed on the support belt

Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Item no. 278046 and 278047

(Artikel Nr., Varenr., Codice, Nr. d'article)

is designed and produced in accordance with the technical documentation and the basic requirements of the:

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In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,