TRANSFER BOARD

Art. 240501, 240502, 240505, 240507, 240701

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

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mobilex.dk/medias/2023-10/ce_transferbraet_14614.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

The Mobiglide Transfer boards are medical devices intended to help disabled persons with the transfer from one sitting surface to another.

Indications

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture. The transfer products are indicated for multiple applications such as: positioning and/or transferring of patients with severely restricted mobility, positioning and/or transferring of patients who are highly sensitive to pain, elevating/lifting patients to different heights etc. Proper application of the transfer products decreases patient discomfort, increases patient safety and makes work of caregivers much easier, eliminating the risk of injuries.

Contraindications

Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Do not climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- It is not recommended to use the transfer board without previous training by a healthcare professional.
- Attempting a transfer without the approval of a professional is not recommended.
- Prior to use always inspect the board for signs of wear.
- Do not use the board if it appears weakened.
- Be sure the board is securely in place before putting weight on it.
- The open space between the two sitting surfaces (e.g. from wheelchair to bed) may not be larger than 20 cm.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Transfer board;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

The Mobiglide transfer boards are made of 6 mm thick white translucent polypropylene, or 6 mm thick grey PVC. The low frictional resistance of the Mobilex transfer boards makes transferring to or from the wheelchair much easier.

Assembly description

This device doesn't require an assembly instruction.

Function description

Using the transfer board

Position one end of the board under the user's body and the other end on the target surface. Make sure that the distance between the two sitting surfaces does not exceed 20 cm, and position the board so that it supports the full body weight during the transfer. The smooth surface of the board is best suited for clothed transfers. For bare-skin transfers a towel can be placed over the board for a smoother transfer.

Accessories

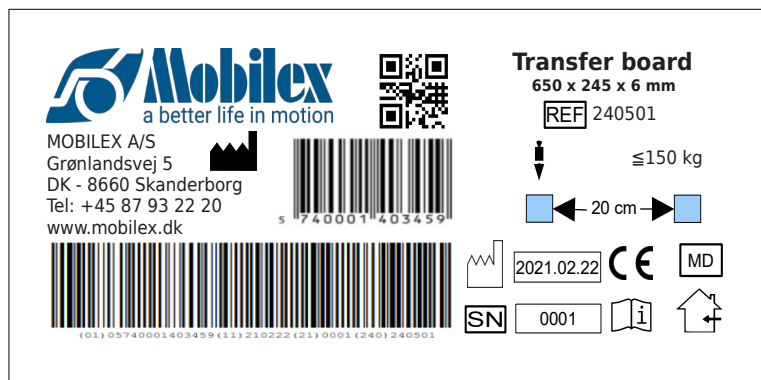
The device can be equipped with the following accessories (not a part of the standard equipment):

- ✓ Anti-slip strips

4. Technical data

Transfer board	Art. 240501	Art. 240502	Art. 240505	Art. 240507	Art. 240701
Dimensions (L x W x H)	650 x 245 x 6 mm	800 x 245 x 6 mm	800 x 245 x 6 mm	800 x 348 x 6 mm	1500 x 700 x 6 mm
Material	PP	PP	PVC	PP	PP
Weight	0,78 kg	0,98 kg	1,54 kg	0,98	4,58 kg
Max. user weight	150 kg	150 kg	130 kg	150 kg	200 kg

Product label

Example of the product label

	QR-code link to homepage		CE marked product
	Manufacturer		Manufacturing date
	Observe the user manual		Max load
	Serial no.		Medical Device
	Item Number		Max distance between surfaces
	Only to be used indoor		

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. The entire product can be disinfected with a commercially available disinfectant. Always follow the disinfectant manufacturer's instructions for use and safety when disinfecting.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ The product should be checked if it has been bent or damaged

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0° C.

Transportation

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
The board is not stable during transfer	The distance between the two sitting surfaces exceeds 20 cm	Make sure that the distance between the two sitting surfaces is not exceeding 20 cm
The transfer of the patient isn't going smoothly	The patient contacts the transfer board with bare skin, which increases friction resulting in non-smooth movement.	Dress the patient or put fabric where patient's body touches the transfer board.

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

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