Mobilex "Mobiglide" Transfer boards

Art. 240501, 240502, 240505, 240507 & 240701

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione

Mobiglide patient transfer board Model 240701
for easy transfer from one surface to another e.g.
between beds or from wheel chair to sitting chair
Made from polypropylene

Mobiglide curved transfer board
Model 240507
Made from polypropylene

Mobiglide transfer board
Model 240505
Made from PVC

Mobiglide transfer board
Model 240502
Made from polypropylene

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Introduction

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Intended use of the device

The Mobilex MobiglideTransfer board is made to help disabled persons with the transfer from one sitting surface to another. The models 240501, 240502, 240507 and 240701 are made of polypropylene and the model 240505 of PVC.

By questions please take contact to your dealer or directly to Mobilex A/S.

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 E-Mail: info@mobilex.dk
--	---

Safety requirements

- Make sure that this user manual is read by all persons using the device. Manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product label.
- Only use the product for the described purpose.

Only use the product for the intended purpose. Do not step up on the device. It is not recommended to use the transfer board without previous training by a healthcare professional. Attempting a transfer without the tavolestance from a professional is not recommended. The boards should not be used to support more than the allowed max. user weight. Prior to use always inspect the board for signs of wear. Do not use the board if it appears weakened. Be sure the board is securely in place before putting weight on it. The open space between the two sitting surfaces (e.g. from wheelchair to bed) may not be larger than 20 cm.

Description

Instructions for use

Position one end of the board under the user's body and the other end on the destination surface. Check that the distance between the two sitting surfaces is not more than 20 cm and position the board so that it supports the full body weight during the transfer. The smooth surface of the board is best suited for clothed transfers. For bare-skin transfers a towel can be placed over the board for a smoother transfer.

Maintenance

Normal dirt can be removed with standard cleaning agents and a soft cloth. Do not use solvents or abrasives.

Accessories

Anti-slip strips are available as accessories, which can be attached to the underside of the board to prevent it from sliding away.

Technical data

Mobilex Transfer board	240501	240502	240505	240507	240701
Dimension (L x W x H)	650 x 245 x 6 mm	800 x 245 x 6 mm	800 x 245 x 6 mm	800 x 348 x 6 mm	1500 x 700 x 6 mm
Material	PP	PP	PVC	PP	PP
Weight	0,78 kg	0,98 kg	1,54 kg	0,98 kg	4,58 kg
Max. user weight	150 kg	150 kg	130 kg	150 kg	200 kg

Product label

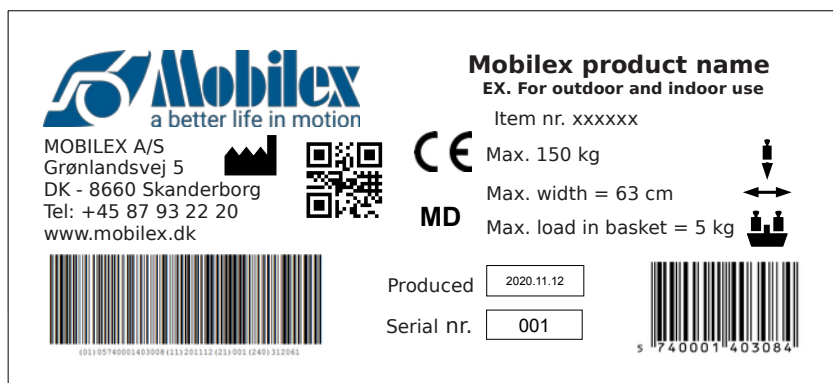
Image of the product label (not original)

Cleaning & maintenance

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded

Recommended service life: Normally 10 years depending on use.

Warning

 Please note that if the product is bent or broken, the product must not be recycled but must be replaced

Reuse

The product is suitable for recycling. Before recycling, where a new user is to take over the product, the product must be checked according to above mentioned advice.

Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,