SLIDING BATH BENCH

Art. 302065

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

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mobilex.dk/medias/2021-12/ce_badetaburet_med_glidesaede_302065_9057.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

The Bath Bench is a medical device intended only to be used by patients that need support because of invalidity during entering or leaving the bathtub and during washing. It allows patients to pass elevated bathtub's side walls with or without the assistance of a helper (depending on user's condition). Physically handicapped persons can use it as stable support and easy access to a bathtub and all human body parts during washing.

Indications

The device is indicated for people with functional limitations, people in wheelchairs and other people with limited mobility of the lumbar spine, knees or hips, and for all those who have difficulties performing everyday personal hygiene in a bathtub. Any use other than stated above is prohibited.

Contraindications

The device is not indicated for those patients who can't maintain proper body balance in a sitting and standing position. The device must not be used if the user has impaired perception. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- Use caution when assembling or adjusting any parts of the device.
- Adjust the legs, so the seat is in horizontal position.
- We recommend to use help of an assistant for getting on and out of the bath bench.
- When applying Bath Bench in to a bathtub - watch carefully for all four feet to lay flat, stable and parallel on the top of floor and bathtub's bottom.
- Always lock seat in position after sliding in or out of the bathtub. Make sure the seat is locked before attempting to sit on it or to get up of it.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Main frame with seat and foldable legs;
- ✓ 1 Backrest;
- ✓ 1 Red, plastic safety handle;
- ✓ 2 Red plastic wingnuts for safety handle assembly;
- ✓ 1 Soap holder;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

The Sliding Bath Bench set consists of base frame made of aluminum, a plastic seat and a backrest. The four foldable legs of base frame are covered with protective rubber feet to be laid down on the floor (2 narrow feet) and bathtub's bottom (2 wide feet). Bath Bench is equipped with safety handle and removable soap holder.

The device consists of the following elements:

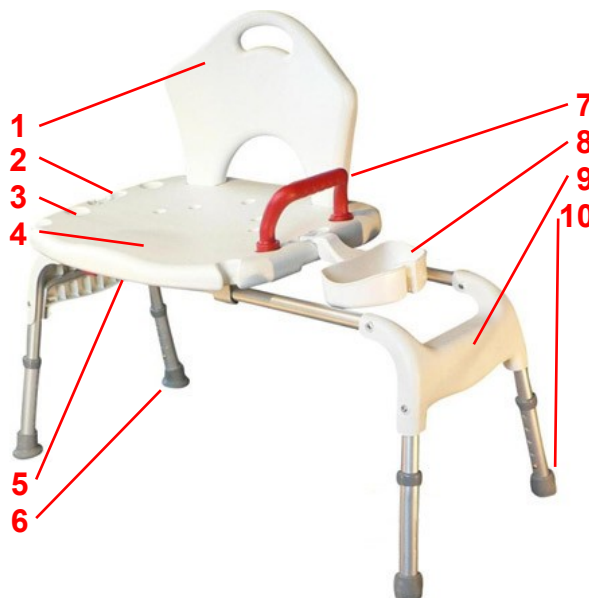
1. Backrest
2. Slot for the soap holder
3. Slot for the safety handle
4. Seat
5. Sliding lock lever (under the seat)
6. Wide rubber foot (for bath)
7. Safety handle
8. Soap holder
9. Leg pivot point with a safety lock
10. Narrow rubber foot

Assembly and initial setup description

1. Unfold all legs according to **Folding the bench** in Function description section.
2. Attach the backrest by pressing it into 2 dedicated rectangular holes located at the back of the seat pad. Check if the backrest is fully inserted and locked in its place safely.
3. Place safety handle's ends into 2 holes on either side of the seat pad. Use 2 red wing nuts to attach the handle to the seat by screwing them on from the bottom.

NOTE: The side on which the handle is attached depends on the conditions of your bathroom and your personal preferences. To ensure safety it is best to attach the handle on whichever side the bench is outside of the bathtub.

4. Attach the soap holder to either side of the seat pad by pushing the end of the holder into the dedicated slot.
5. It is possible to switch the bottom part of the legs to the other side to allow the bath bench to face the opposite direction. To do so, simply press on the button located on each leg, and pull them out. Then insert the legs on opposite sides. When the bench is placed wide feet should be inside the bath and narrow feet on the bathroom floor.
6. The height of the bench should be adjusted according to **Height adjustment** in Function description section.

Function description

General use

The Bath Bench is to be placed with the wider feet in the bath and narrow ones outside of it on the floor. Before use the bench and the seat must be level.

Folding the bath bench

The legs of the bench can be folded inwards by pressing the buttons at the leg's pivot points and folding them. To unfold the legs, pull them outwards until the safety lock "clicks", which indicates the legs are locked in position.

Adjusting the height of the bath bench

The height of the bench is adjusted by changing the length of the legs. To adjust a leg, push in the pin located on the leg on the inner side of the bench and slide the outer tube to a desired setting so that the pin will pop out. The legs should be adjusted to account for the difference of height between the floor and the bath floor, so the bench will still be level and the seat pad will have enough space to slide over the wall of the bath.

WARNING! Always before usage ensure that legs are fully opened and locked in that position.

Sliding seat pad

The sliding seat pad is used to easily get in and out of the bath. First unlock the sliding lock by pulling the red lever beneath the seat pad, then move with legs until in correct position. Release the lever to lock the seat pad in place.

Accessories

There are no accessories provided for this device.

4. Technical data

Sliding bath bench	Art. 302065
Overall width	107 - 111 cm
Overall length	49 - 53 cm
Overall Height (adjustable)	89,6 - 99,5 cm
Seat width	50 cm (with handle) / 57 cm (without handle)
Seat depth	39,5 cm
Under seat height (max. height of the edge of the tub)	45,5 - 55,5 cm
Weight of the product	6,5 kg
Max. user weight	130 kg
Backrest height	33 cm

Product label

Example of the product label

	QR-code link to homepage		CE marked product
	Manufacturer		Manufacturing date
	Observe the user manual		Max load
	Serial no.		Medical Device
	Item Number		To be used indoor and outdoor

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. The entire product can be disinfected with a commercially available disinfectant. Always follow the disinfectant manufacturer's instructions for use and safety when disinfecting.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
The seat is not in horizontal position	Legs are not adjusted to correct height	Readjust the legs length so the seat is in horizontal position when bench is placed in bathtub.
Backrest is not stable	Backrest is not pressed all the way in to dedicated holes	Press backrest all the way in to it's place.
Safety handle isn't stable	The wing nuts under the handle aren't fixed	Tighten the wing nuts.
The frame isn't stable	Legs are not fully opened and locked.	Pull the legs outward until you'll hear the "click" sound of their safety lock.

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

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