Sliding Sheets

Item no. 278011, 278012, 278013, 278015, 278016 & 278023

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1. Introduction

This user manual contains a description of the product and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Indications

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture. The transfer products are indicated for multiple applications involving people with mobility limitations, who require assistance during: positioning and/or transferring of patients with severely restricted mobility, positioning and/or transferring of patients who are highly sensitive to pain., elevating/lifting patients to different heights etc. Proper application of the transfer products decreases patient discomfort, increases patient safety and makes work of caregivers much easier, eliminating the risk of injuries.

Contraindications

Contraindications cannot be tied to one, specific clinical picture. The choice of transfer device appropriate for a given clinical picture and task to be completed must be always made with the help of a specialist physician or physiotherapist. Observe the safety information in chapter 2.

Warning

Sliding Sheets are designed for low friction positioning or repositioning directly on the surface and **not for lifting.**

By questions please take contact to your dealer or directly to Mobilex A/S

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 E-Mail: info@mobilex.dk
--	---

2. Security instructions

- Make sure that this user manual is read by all persons using the device. Manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- Follow the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes on the product, unless you have the manufacturers written acceptance for such modifications.
- Always check that the label is readable and that the product is in good condition without signs of tear and wear such as loose seams, frays and material inconsistency.
- Do not leave the product unattended on the floor, a person may accidentally step on it with the risk of injury.

Warning! Do not use the product if defects, errors or deterioration is detected. Contact your dealer immediately.

3. Description

Product description

Applying the Tube Sliding Sheet

- The patient can be in lying or sitting position.
- Decide the sliding direction before introducing the sliding sheet under the patient:
 - a) Forward or backward movement; Fold the Sliding Sheet and introduce it under the patient in forward/backward direction and roll it out in the desired position.
 - b) Side-to-side movement: fold the Sliding Sheet, introduce it under the patient in side to side direction, and roll it out in the desired position.

- Carefully perform repositioning in the desired direction whilst keeping eye on the patient position and comfort.
- After performing the repositioning, remove the Sliding Sheet by pulling it carefully away.

Applying the Flat Sliding Sheet with handles

- The patient can be in lying or sitting position. Lying for lateral transfer bed to bed.
- Fold the Sliding Sheet, introduce it under the patient, and roll it out in the desired position.
- The patient may now be moved in any direction or if lateral transfer side to side.
- Carefully perform repositioning into desired position by holding the handles of the Flat sliding Sheet enclosing the patient within, while keeping eye on the patient position and comfort.
- After performing the repositioning, remove the Sliding Sheet by pulling it carefully away.

Applying the Flat Sliding Sheets in pairs

- The patient can be in lying or sitting position, bed or wheelchair/chair.
- Two Flat sliding sheets can be used for easier positioning a lifting sling for lifting the patient.
- Introduce both Sliding Sheets under/behind the patient, and roll it out/down in the desired position.
- Using the top Sliding Sheet you can move the patient as it slides against the bottom Sliding Sheet and creates a smooth and easy transfer for both the patient and the carer.
- Place the sling between the two Sliding Sheets and slide/pull it in position down behind the patients back.
- When the sling is in the desired position, carefully remove the Sliding Sheets.


Accessories

There are no accessories for the Mobilex slide sheets.

4. Technical data

Item no.	Description	Material	Dimensions	Max. user weight
278011	Tube Sliding Sheet	Nylon (PA)	70 x 68 cm	250 kg
278012	Tube Sliding Sheet	Nylon (PA)	140 x 68 cm	250 kg
278013	Tube Sliding Sheet	Nylon (PA)	200 x 68 cm	250 kg
278015	Flat Sliding Sheet (to be used in pairs)	Nylon (PA)	150 x 100 cm	250 kg
278016	Flat Sliding Sheet (to be used in pairs)	Nylon (PA)	200 x 100 cm	250 kg
278023	Flat Sliding Sheet with handles	Nylon (PA)	200 x 100 cm	250 kg

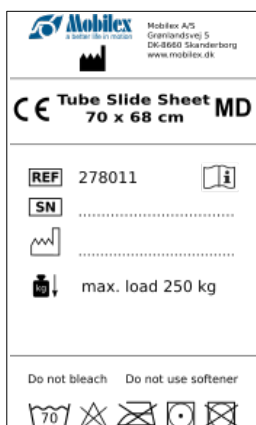Product label

The product label is sewed on the Sliding Sheet

	QR-code link to homepage		Manufacturing date
	Manufacturer		Medical Device
	Observe the user manual		serial number
	CE marked product		Item number
	Drip-dry		Medium Hot-iron
	washing temperatur		Do not tumble
	Handwash		Max width

Overview icon meaning

5. Maintenance and care

Before and after every use

- Always perform a risk assessment before starting the specific repositioning session in order to make sure the product model is correct and that the procedure is fully understood.
- Before using the product, read this user manual and the user manual for any additional equipment used.
- When soiled or stained the product must be replaced or washed according to the product label.
- After use check the product according to step 1 to 3 above and store in a dry area.
- The product shall regularly be checked by authorized personnel and always after washing.

Recommended service life

Expected service life is 2 years; however, this will be subject to the actual conditions during use.

Storage

The product should be stored in a dry and clean environment. The shelf lifetime is 3 years in original packaging.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

6. Trouble shooting

If reduced sliding effect, please check if the washing or drying temperature has been too high or if the product has actually reached its lifetime. In doubt please contact your dealer who supplied the product. Always follow the instructions on the product label and check the product for deterioration.

7. Disposal of the product

The product may not be disposed of with the household waste but must be brought to the local recycling site.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices. The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

Tube Slide Sheet - item no. 278011, 278012, 278013

Flat Slide Sheet - item no. 278015, 278016

Flat Slide Sheet with handles - item no. 278023

Harmonized norms used during conformity estimation:

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In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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