BUFFALO HD Rollator

Art. 312305, 312307, 312385 & 312387

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions for use contain a description and important information about the use of the medical product. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the product. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the product

The Mobilex product is medical device which has been designed to increase the mobility of persons who have difficulties standing or walking. The rollator enables those persons to regain the ability to perform daily activities. The rollator is suitable for persons who as a result of various diseases need stable walking support. The rollator should only be used as a support for standing, walking and sitting. Any other use is prohibited.

Indications

The rollator is a medical device designed to help people with functional limitations. The indications for use are diseases that cause mobility limitations and disabilities, obstructing independent sitting, standing and walking. Therefore, there is a need for an additional stable support when performing the above activity. Indications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status by professional physician or physical therapist, but main target user groups are elderly people, persons with paresis, rheumatic diseases, and other people with limited mobility of the lumbar spine, knees or hips. Using the product for any purpose other than those described above is prohibited.

Contraindications

Rollators are not indicated for those persons who can't maintain proper body balance in a standing position. The device must not be used if the user has problems with impaired perception. Contraindications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the product, please contact the dealer or manufacturer:

Mobilex A/S	Phone:	+45 87 93 22 20
Grønlandsvej 5	Telefax:	+45 87 93 17 77
DK-8660 Skanderborg	Email:	info@mobilex.dk
Denmark		

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't stand on to the device.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- When adjusting the device the user must take care not to squeeze the fingers or other body parts.
- Always check before use that the wheel brakes function perfectly.
- Before using the rollator, make sure it is fully opened and the anti-folding lock is active.
- The rollator should only be used on a flat stable surface.
- The seat may only be used when the brakes have been firmly locked.
- Both brakes should always be activated simultaneously.
- Do not exceed the maximum weight capacity of 200 kg.
- Max. load of the shopping bag is 5 kg.
- It is not allowed to carry an extra person on the seat.
- Both brakes must be always active if the rollator is not in use.
- All 4 wheels must always be in contact with the ground.
- The rollator is not intended to be moved while seated. The seat may only be used when the rollator is standing still with the brakes on.

Parts supplied

The device is supplied in a carton box, folded for easier transportation.
Each product package should contain:

- ✓ 1 rollator including: a mounted seat, back belt, crutch holder and shopping bag
- ✓ 1 user manual;

Preliminary check

Check the contents of the box against the above list. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

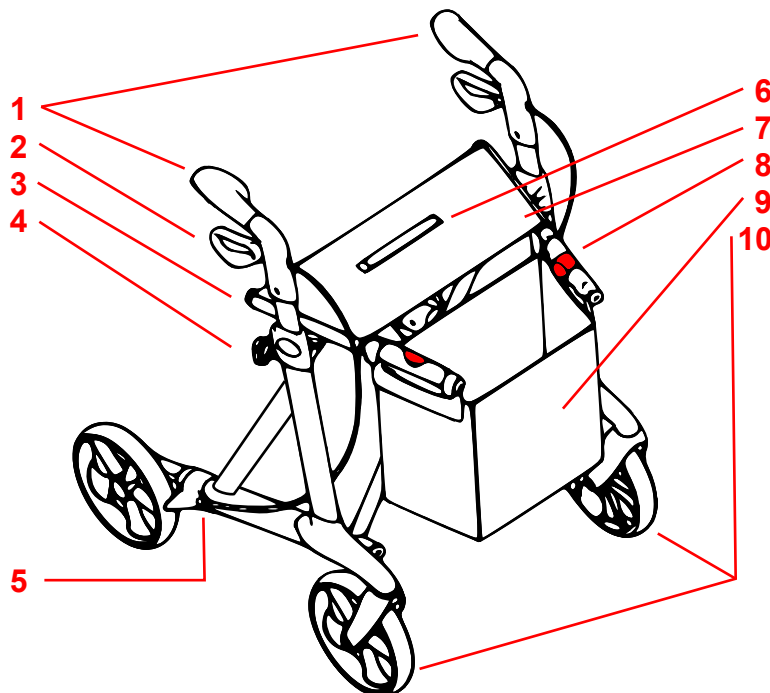
3.Description

Product description

The Buffalo rollator has a powder coated reinforced frame with 4 wheels. The light-weight frame makes folding it easy and offers simple storage and transport. The rollator runs easily and is sufficiently stable. The rollator is delivered with a mounted seat, backbelt, crutch holder and shopping bag.

The rollator consists of:

1. Push handles
2. Brake handles
3. Tubes under seat (frame opens when you press from both sides)
4. Push handle height adjustment knobs
5. Brakes
6. Folding strap (pull here to fold the rollator)
7. Seat
8. Fold lock release button
9. Shopping bag
10. Wheels

Function description

Unfolding the rollator

Press the red (fold lock release) button on the seat tube (see picture on the right). Press both seat tubes down until the frame unfolds. When the frame "clicks" the rollator is locked in opened position.

To make sure that the frame has been fully opened try to press the rollator sides together. Check that the knobs for height adjustment have been firmly tightened. Check that both handles are at the same height and stable.

WARNING! Use of a defective rollator is strictly forbidden. Always check before use the function of all parts of the rollator. If the above recommendations are not followed, injuries may occur.

Folding the rollator

To fold the rollator, pull up the strap on the seat until the fold lock comes together and "clicks".

WARNING! Both handles should always be adjusted to the same height. The correct height of the handles should always be the elbow height of the user when he stands with his arms hanging loose.

Height adjustment of push handles

To adjust the height of the push handles loosen the height adjustment knob by turning it counter-clockwise and adjust the height. Re-tighten the knob by turning it clockwise and repeat for the other handle. Make sure both handles are at the same height.

Using the Brakes

The brakes are gradually activated by pulling the brake handles up. To stop the rollator completely both brake handles must be fully pulled up. To move the rollator again simply deactivate the brakes by letting go of the brake handles.

WARNING! Always activate the brakes when using it on an inclined surface.

Parking brakes

Before activating parking brakes, make sure the rollator isn't moving. To park the rollator push the brake handles downwards until they "click". To deactivate parking brakes pull the brake handles up.

WARNING! Always activate parking brakes when the rollator is not in use and/or on an inclined surface. Before deactivating the parking brakes, stand behind the back of the rollator and grab both push handles firmly.

WARNING! Always activate the parking brakes before sitting on the rollator. When seated, never lean backwards. It is recommended to sit only at the front part of the seat. Do not sit down on the bag.

Assembly

This device doesn't need assembly instructions.

Accessories


The rollator can be equipped with the following accessories (not a part of the standard equipment):

- Rigid seat with upholstery
- Tray
- Speed reduction brake
- Soft wheels
- Mountable and foldable umbrella

4. Technical Data

Rollator Buffalo HD	Standard Item nr. 312305	Small Item nr. 312307	Standard with SOFT wheels Item nr. 312385	Small with SOFT wheels Item nr. 312387
Height of the push handles (adjustable)	74 - 102 cm	66 - 86 cm	74 - 102 cm	66 - 86 cm
Recommended user heights	150 - 200 cm	125 - 170 cm	150 - 200 cm	125 - 170 cm
Seat height	62 cm	55 cm	62 cm	55 cm
Total width opened / folded	68 / 22,5 cm			
Inner width between push handles	56 cm			
Length (depth)	66 cm			
Weight (+ bag 380 g)	7,8 kg	7,5 kg	7,9 kg	7,6 kg
Max. load (rollator / bag)	200 kg / 5 kg			
Brakes	2 lockable with parking function			
Wheel sizes	Front and back 200 x 35 mm			

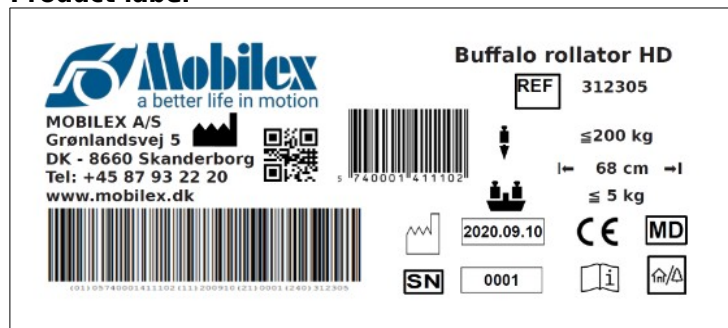Product label

Example of product label

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device

Meaning of icons

5. Care & Maintenance

Normal dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

Wheels can be cleaned with hot water with a mild cleaning agent. After cleaning the wheel bearings should be lubricated with oil periodically or when the wheels start squeaking.

The product basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General re-tightening of the product

Expected service life: Normally 5 years depending on use.

Storage: We recommend storage in a dry place at a temperature above 0, away from intense sunlight.

Transportation

Before transportation fold the rollator.

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, it must be checked according to point 5.

6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
The rollator is difficult to manouver or push	Dirt between wheels and forks	Remove dirt or hair from between wheels and forks
The brakes do not work or work only on one side	The brake shoes are worn, or the brakes are not adjusted correctly	Adjust the brakes. Replace the brake shoes if needed
The seat does not fit in when the rollator is unfolded	The locking device is in a wrong position or defective	Check that the locking device is in a correct position. Replace it if needed

7. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark, **SRN: DK-MF-000021885**