Wheelchair cushions

Art. 278372, 278373, 27840N, 278402, 27840P, 27842N, 278422, 27842P, 27845N, 278452, 27845P, 27847N, 27847P, 27850N, 27850P, 278552, 27840H, 27842H, 27844H, 27848H, 27851H, 27854H, 27857H, 27840Q, 278407, 27840R, 27842Q, 278427, 27842R, 27842S, 27845Q, 278457, 27845R, 27845S, 27845F, 27847Q, 27847R, 27850Q, 27850R, 27855R, 27840C, 27842C, 27845C

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur


Standard wheelchair cushions are made of high quality cold foam. Available in the respective sizes with both PU foam and viscoelastic foam with high density.


Polyester or incontinence cover (5 or 6 cm high)

Zipper on the back to easily remove the cover for washing. Rounded front to reduce pressure on the lower thigh.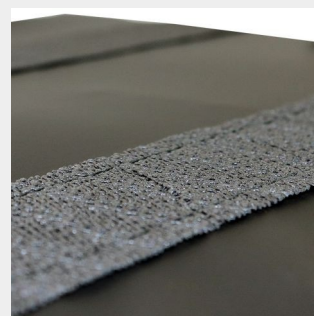

Bottom with an anti-slip material so it can be used on all wheelchairs regardless of the position of the velcro

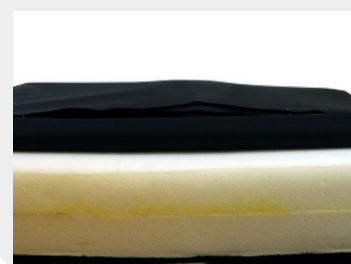The cushions have a standard height of 5 cm and are available in 42-48 cm depth (adjustable), available in 6 sizes (widths) from 40 to 50 cm. Tested for 125kg.

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

The wheelchair cushions are medical devices designed to provide comfort to wheelchair users and reduce discomfort when sitting in a wheelchair for extended periods of time. The wheelchair cushions are used to greatly reduce the risk of developing bedsores. Any other use is prohibited.

Indications

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture. The Wheelchair cushions are indicated for people who spend a significant amount of time in wheelchairs and for those at risk of developing bedsores. Indications also depend on individual evaluation of potential user's health status by professional physician or physical therapist, but main target user groups are elderly people, persons with paresis, rheumatic diseases, and other people with limited mobility of the lumbar spine, knees or hips. Using the product for any purpose other than those described above is prohibited.

Contraindications

Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product is suitable for indoor and outdoor use and can be used in home care, in health and care facilities and in hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S	Phone:	+45 87 93 22 20
Grønlandsvej 5	Telefax:	+45 87 93 17 77
DK-8660 Skanderborg	Email:	info@mobilex.dk
Denmark		

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- Always check that the product is in good condition without signs of tear and wear such as loose seams, frays and material inconsistency.
- Do not leave the product unattended on the floor, a person may accidentally step on it with the risk of injury.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Wheelchair cushion;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

Standard wheelchair cushions

Wheelchair cushions are made of high quality cold foam. The bottom of the cushion is a non-slip material sewn on either as a whole surface or in two bands, so it can be used on all wheelchairs, regardless of the position of the velcro. The front of the cushion is rounded to reduce pressure on the lower thigh. The cushions are equipped with a zipper on the back and 10 cm down both sides, so you can easily remove the cover for washing.

The cover is sewn from polyester or PU coated fabric, which ensures great durability, as well as good stretchability.

The cushions have a standard height of 5 cm and are available in 42-48 cm depth (adjustable), available in 6 sizes (widths) from 40 to 50 cm. The cushions are also available in the respective sizes with both PU foam and viscoelastic foam with high density.

The standard cushions are available in black polyester or incontinence cover (5 or 6 cm height). All cushions are equipped with a zipper on the back and an anti-slip bottom.

Universal and adjustable wheelchair cushion

Universal wheelchair cushion made of cold foam with black incontinence cover and adjustable seat depth. The cover is equipped with Velcro and can be easily opened. The cushion is equipped with grooves for depth adjustment of the cushion. The foam is cut to the desired depth. The cover can be attached to the chosen depth with Velcro. The cushion is supplied as standard with our wheelchair "Dolphin" and "Barracuda".

Cushions are available for seat widths from 40 to 51 cm. Cushion depth can be reduced from 48 cm to 42 cm.

Wheelchair cushion with 2 layers of foam (viscoelastic and cold foam)

The wheelchair cushion is made of a layer of viscoelastic foam on top and a regular cold foam on the bottom. Supplied complete with incontinence cover and anti-slip underside. Available in 5 widths from 40 - 50 cm and in 2 depths.

Assembly description

This device doesn't require an assembly instruction.

Accessories

There are no accessories provided for this device.

4. Technical data

Item no.	Description	Material	Dimensions	Max. User weight
All	See page 16	See page 16	See page 16	See page 16

Product label

Example of product label sewed on the wheelchair cushion

Meaning of the icons

	QR-code link to homepage		CE marked product		Drip-dry
	Manufacturer		Manufacturing date		Medium ironing temperature
	Observe the user manual		Max load		Washing temperature
	Batch number		Medical Device		Do not tumble dry
	Item Number		Only to be used indoor		Handwash

Example of product label on the box

5. Care and maintenance

The outer pillow cover is removable and machine washable in 95°C water. Please refer to the washing instructions on the label. The zipper of the pillowcase should be closed when washing. Do not use a tumble dryer to dry the pillowcase but air dry naturally.

The cushion itself cannot be soaked and washed in water, as this will affect its functioning. Any water that gets inside the foam will be trapped in it and will be very difficult to remove. Use only damp cloths to clean the pillow and dry naturally after cleaning. Do not wash, iron, or blow dry with hot air. This can damage the fabric of the product.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Check the fabric for wear and replace the product if necessary
- ✓ When soiled or stained, the product must be replaced or washed according to the product label

Recommended service life: Normally 5 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

The pillow allows for easy transport. We recommend keeping the pillow packaging for ease of transport and storage. All cushions can ONLY be rolled and not folded or bent. Failure to do so may cause partial damage to the foam.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

Problems	Possible causes	Solutions
Comfort of the cushion is reduced	Washing or drying at too high temperatures or the product reached its lifetime	Contact the dealer who supplied the product

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at mobilex.dk/medias/2022-04/ce_pude_med_sort_inkontinensbetaek_278372_27845c_10052.pdf or via the QR code on the right.

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Technical data:

Standard wheelchair cushions made of PU foam

Item no.	Description	Cover	W x D x H
278372	Wheelchair cushion made of PU foam (H=50 mm) with anti-slip bottom	Incontinence	37,5 x 40,0 x 5 cm
278373		Incontinence	37,5 x 43,0 x 5 cm
27840N		Incontinence	40,0 x 40,0 x 5 cm
278402		Incontinence	40,0 x 43,0 x 5 cm
27840P		Incontinence	40,0 x 45,0 x 5 cm
27842N		Incontinence	42,5 x 40,0 x 5 cm
278422		Incontinence	42,5 x 43,0 x 5 cm
27842P		Incontinence	42,5 x 45,0 x 5 cm
27845N		Incontinence	45,0 x 40,0 x 5 cm
278452		Incontinence	45,0 x 43,0 x 5 cm
27845P		Incontinence	45,0 x 45,0 x 5 cm
27847N		Incontinence	47,5 x 42,0 x 5 cm
27847P		Incontinence	47,5 x 48,0 x 5 cm
27850N		Incontinence	50,0 x 42,0 x 5 cm
27850P		Incontinence	50,0 x 48,0 x 5 cm
278552		Incontinence	55,0 x 48,0 x 5 cm
27840H	Universal wheelchair cushion made of PU foam with anti-slip bottom Cushion is adjustable in depth	Incontinence	40,0 x 42-48 x 5 cm
27842H		Incontinence	42,0 x 42-48 x 5 cm
27844H		Incontinence	44,0 x 42-48 x 5 cm
27848H		Incontinence	48,0 x 42-48 x 5 cm
27851H		Incontinence	51,0 x 42-48 x 5 cm
27854H		Incontinence	54,0 x 42-48 x 5 cm
27857H		Incontinence	57,0 x 42-48 x 5 cm

Two-layer standard wheelchair cushion (viscoelastic + PU)

Item no.	Description	Cover	W x D x H
27840Q	Two-layer wheelchair cushion with one layer made of viscoelastic foam (top) and one layer made of PU foam (bottom) with anti-slip bottom	Incontinence	40,0 x 40,0 x 8 cm
278407		Incontinence	40,0 x 43,0 x 8 cm
27840R		Incontinence	40,0 x 45,0 x 8 cm
27842Q		Incontinence	42,5 x 40,0 x 8 cm
278427		Incontinence	42,5 x 43,0 x 8 cm
27842R		Incontinence	42,5 x 45,0 x 8 cm
27842S		Incontinence	42,5 x 48,0 x 8 cm
27845Q		Incontinence	45,0 x 40,0 x 8 cm
278457		Incontinence	45,0 x 43,0 x 8 cm
27845R		Incontinence	45,0 x 45,0 x 8 cm
27845S		Incontinence	45,0 x 48,0 x 8 cm
27845F		Incontinence	45,0 x 50,0 x 8 cm
27847Q		Incontinence	47,5 x 42,0 x 9 cm
27847R		Incontinence	47,5 x 48,0 x 9 cm
27850Q		Incontinence	50,0 x 42,0 x 9 cm
27850R		Incontinence	50,0 x 48,0 x 9 cm
27855R		Incontinence	55,0 x 48,0 x 9 cm

Standard wheelchair cushion made of high density viscoelastic foam

Item no.	Description	Cover	W x D x H
27840C	Wheelchar cushion made of high density viscoelastic foam with anti-slip bottom	Incontinence	40,5 x 43 x 5 cm
27842C		Incontinence	42,5 x 43 x 5 cm
27845C		Incontinence	45,0 x 43 x 5 cm