**Rollator PANTHER**  
Art. 318001 & 318002

**Montage- und Gebrauchsanleitung**  
**Montage- og brugervejledning**  
**User Manual**  
**Manuale d'uso e manutenzione**  
**Installations- och bruksanvisning**

Panther lightweight rollator has a blue aluminium frame  
For both indoor and outdoor use  
Available in 2 sizes  
Standard (62 cm)  
Small (55 cm)  
Tested for 150 kg

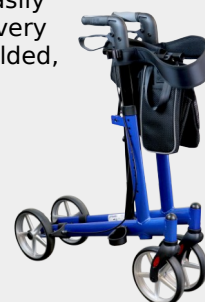
Height adjustable  
Push handles.  
No tools needed  
Integrated tilt  
pedal and 2  
lockable brakes

Removable bag with  
two handles that can also  
be used for shopping

The rollator folds easily  
sideways Takes up very  
little space when folded,  
for easy transport

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## 1. Introduction

Thank you for buying a high-quality Mobilex product. This user manual contains a detailed product description and important guidelines for correct use of the product. You must read the instructions carefully before using the product. Please always observe all guidelines, in particular the security instructions, in order to use the product safely.

### Intended use of the device

The Panther rollator is a medical device which has been designed to increase the mobility of persons who have difficulties standing or walking. The rollator enables those persons to regain the ability to perform daily activities. The rollator is suitable for persons who as a result of various diseases need stable walking support. The rollator should only be used as a support for standing, walking and sitting. Any other use is prohibited.

### Indications

The rollator is a medical device, the intended use of which is to assist disabled people when walking. The indications for use are diseases that cause mobility limitations and disabilities, obstructing independent sitting, standing and walking. Therefore, there is a need for an additional stable support when performing the above activity. Indications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status by professional physician or physical therapist, but main target user groups are elderly people, persons with paresis, rheumatic diseases, and other people with limited mobility of the lumbar spine, knees or hips. Using the product for any purpose other than those described above is prohibited.

### Contraindications

Rollators are not indicated for those persons who can't maintain proper body balance in a standing position. The device must not be used if the user has problems with impaired perception. Contraindications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

If you have any questions, please contact your dealer or Mobilex A/S

<b>Mobilex A/S</b> Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
------------------------------------------------------------------------	------------------------------------------------------------------------------

## 2. Security Instructions

- Use only the product if it is technically flawless
- In case of any product damage or faults please contact your dealer immediately
- Observe the data on the product label
- Use the product only for the described purpose
- Please notice that there is a risk of injury at all joints on the rollator
- Always check before use that the wheel brakes function perfectly
- Use the rollator only when it has been fully unfolded and the locking device has been activated
- The rollator should only be used on a flat stable surface, indoors or outdoors
- The seat may only be used when the brakes are firmly locked
- Both brakes should always be activated or locked simultaneously
- Max. load of the rollator is 150 kg in total
- Max. load of the shopping bag is 5 kg
- It is not allowed to carry an extra person on the seat
- The product must not be changed without the manufacturers written permission
- The rollator must always be braked when not in use

## 3. Description

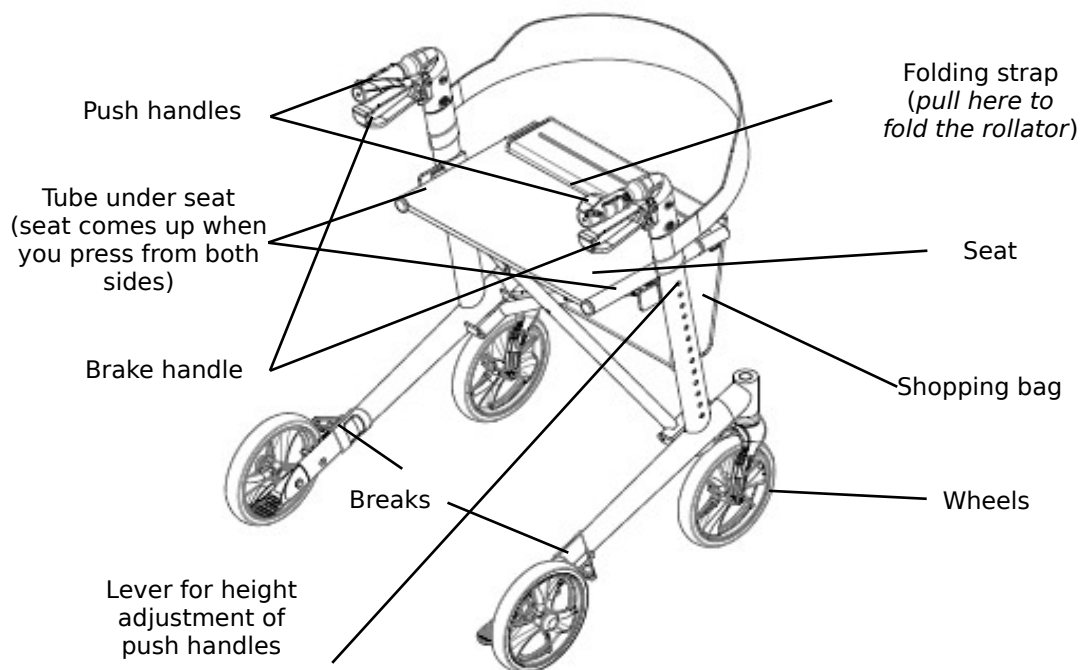
The Panther rollator has a powder coated frame and 4 wheels. The light-weight frame makes it easy to fold the rollator and offers easy storage and transport. The rollator runs easily and gives a stable support. The height of the push handles can easily be adjusted. The rollator is delivered with a mounted seat, back strap, crutch holder and a shopping bag.

### Parts supplied and mounting instructions

The Panther rollator is delivered in a carton with the following content:

#### Parts supplied:

- ✓ 1 rollator inclusive mounted seat, back strap, crutch holder and a shopping bag
- ✓ This user manual

### Control upon delivery

After unpacking the rollator please check if all parts shown in the drawing have been delivered. If any parts are missing please contact your dealer immediately.

### Unfolding the rollator

- To be able to unfold the walker, the black locking fuse must be secured by tilting it off one of the pipes.
- Press down both tubes under the seat until the frame unfolds and you can hear a "click". Now the locking device is in a fixed position.
- To be sure that the frame has been fully unfolded check that it is probably locked.
- Attach the strap which connects both push handles (see photo at the right). When the rollator has been unfolded, you must check that the strap is blocked.
- Check that the levers for height adjustment have been firmly tightened.
- Check that both handles are at the same height and stable.

*Use of a defective rollator is strictly forbidden.*

*Always check before use the function of all parts of the rollator. If the above recommendations are not followed, injuries may occur.*

### Folding the rollator

- Pull the strap at the seat until the frame folds together.
- Now the rollator is folded together.

### Height adjustment of push handles

- Unfold the rollator
- Press the adjustment button.
- Adjust the height
- Make sure that the adjustment button pops out again.
- Make sure both push handles are adjusted in the same height.

*Adjust the height of the push handles so that the handles are at wrist height when the user stands next to the rollator with arms vertically downwards. This means that the arms have an angle of 20°- 30° when using the rollator.*

## Brake function

- To brake you must pull both brake handles towards yourself
- The more you pull the higher the braking capacity
- To ensure that the rollator remains STOPPED you must keep the brake handles pulled To make the rollator MOVE you must let go of the brake handles whereby the braking will stop

*Never release the brakes when seated on the rollator!  
Always activate the brakes when not using the rollator!  
When placing the rollator on a non-plane surface you must always activate the brakes!*

## Activating and deactivating brakes

- To stop the rollator pull the brake handles upwards
- To park the rollator push the brake handles downwards until a click is heard
- Do not sit on the rollator unless the brakes are in a locked position
- Before setting the rollator into motion you must stand up in the walking direction and release the brake handles by pulling them upwards

*The brakes must always be in a locked position when the rollator is parked!  
Never use the seat if the brakes are not locked! When seated, never lean backwards. It is recommended to sit only at the front part of the seat.*

## Function description

As the frame can be closed the rollator can fold, and as a result it requires very little space. Push handles can be freely adjusted to fit the size of your body. The seat allows you to rest.

## Accessories

For the Panther Rollator various accessories are available such as crutch holders, speed reducing brake etc. Please contact your dealer.

## Transport recommendations

It is recommended to transport the rollator in the folded position. Its construction allows easy unfolding and folding. You should check that the rollator is in a stable position during transport to avoid any deformation.

## Use

Stand in the middle behind the back wheels. Grab the push-handles and smoothly start walking. Due to the special design of the brakes, only a minor effort is required to use the brakes.

## 4. Technical data

<b>Rollator Panther</b>	<b>Standard 318001</b>	<b>Small 318002</b>
<i>Height of the push handles (adjustable)</i>	78 - 101 cm	71 - 93 cm
<i>Recommended user heights</i>	150 - 195 cm	140 - 187 cm
<i>Seat height</i>	62 cm	55 cm
<i>Total width opened / folded</i>	62 / 22 cm	62 / 22 cm
<i>Inner width between push handles</i>	43 cm	43 cm
<i>Length (depth)</i>	75,5 cm	75,5 cm
<i>Weight (+ bag 380 g)</i>	6,73 kg	6,25 kg
<i>Max. load (rollator / bag)</i>	150 kg / 5 kg	150 kg / 5 kg
<i>Brakes</i>	2 lockable with parking function	2 lockable with parking function
<i>Wheel sizes</i>	Front and back 200 x 35 mm	Front and back 200 x 35 mm

### Product label

A product label is placed on the lower frame

Product label for 318001

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device
	Only to be used indoor		

Overview icon meaning

### 5. Maintenance and care

Normal dirt on metal and plastic parts can be removed with normal cleaning agents. Check the contents of the cleaning agents and use only products suitable for cleaning and disinfection. Do not use solvents or abrasive cleaning agents

Despite the solid structure and use of resistant and anti-corrosive materials the rollator is subject to wear. It is therefore recommended to check the rollator regularly (once a year as a minimum). Such a check should comprise: A visual control of all parts of the frame: if they have been bent, damaged, worn or corroded and in particular a control of the following

- ✓ Rollator frame
- ✓ Push bars, handles and strap
- ✓ Brake handles and brake cables
- ✓ Forks and front wheels
- ✓ Braked back wheels
- ✓ Seat and locking device
- ✓ Shopping bag
- ✓ Check if any mounting parts are missing (e.g. nuts, screws, rivets ...) and if they function and are firmly tightened
- ✓ Check the function of the brakes
  - by lowering the speed
  - by braking the wheels

### Cleaning the wheels

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

**Recommended service life:** Normally 10 years depending on use.

### Storage

We recommend storage in a dry place.

### Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

## 6. Troubleshooting

<i>Problems</i>	<i>Possible Causes</i>	<i>Solutions</i>
<i>The rollator is difficult to guide or push</i>	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
<i>The brakes do not work or work only in one side</i>	The brake shoes are worn, or the brakes are not adjusted correctly	Adjust the brakes. Replace the brake shoes if needed (→ service centre)
<i>The seat does not fit when the rollator is unfolded</i>	The locking device is in a wrong position or defective	Check that the locking device is in a correct position. Replace it if needed (→ service centre)

## 7. Disposal of the product

The product may not be disposed of with household waste but must be brought to the local recycling site.

## 8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

**PRODUCT LIST**

**Panther rollator - item no. 318001 +02**

**ACCESSORIES LIST**

Item no.	Accessories item no.
318011	Tray for Panther rollator

Harmonized norms used during conformity estimation:  
PN-EN ISO 11199-2:2005; EN ISO 14971:2012, EN 1041:2009

**In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted**

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Vid upptäckt av material- och produktionsfel, vänlig kontakta återförsäljaren omedelbart eller direkt till Mobilex A/S.