DOLPHIN & BARRACUDA

Art. 271340 - 271357, 271440 - 271457, 271840 - 271851 &
272040 - 272051

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

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mobilex.dk/medias/2021-11/ce_barracuda_koerestol_271840_271851_8780.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

The intended use of the wheelchair is to allow the user to move around in a sitting position, independently or with the help of an attendant. Using the wheelchair for purposes other than those indicated above, is prohibited.

Indications

Manual wheelchairs are indicated for people with mobility limitations who have problems with walking, standing or sitting without stable support.

Contraindications

The wheelchair should not be used by people who cannot maintain a stable sitting position. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S	Phone:	+45 87 93 22 20
Grønlandsvej 5	Telefax:	+45 87 93 17 77
DK-8660 Skanderborg	Email:	info@mobilex.dk
Denmark		

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- The device must only be used on stable surface.
- When adjusting the device the user must take care not to squeeze the fingers or other body parts between components.
- Ensure that all 4 wheels are always in contact with the surface during use. This will ensure proper wheelchair balance and will help avoid accidents.
- Using the wheelchair after exposing it to direct and prolonged contact with the sun may lead to skin burns.
- It is not recommended to use the wheelchair on the beach, in mud or in extreme weather conditions.
- The attendant of the wheelchair user should be physically capable and completely healthy.
- Avoid leaving the wheelchair near sources of heat and open flames (campfires, stoves). Smoking is also not recommended during use as the wheelchair is not fire resistant.

Parts supplied

The device is supplied in a carton box, disassembled for easier transportation. The contents of the package are:

- ✓ 1 Main frame with installed: upholstered back and seat, 2 armrests, 2 Detachable footrests;
- 2 drive wheels, 2 brakes, 2 steering wheels, 2 quick release axles, 1 ✓ 1 Seat cushion;
- step pedal and optionally: 2 anti-tips, 2 drum brakes; ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

The device provides:

- A comfortable transportation and storage thanks to its crossed frame structure and wheels with a quick-release axle
- Easiness of maneuvering
- Easy transfer to - and out of the wheelchair
- The durability and the solidity of the structure

The device consists of the following elements:

1. Push handles - enables the attendant to maneuver the wheelchair.
2. Armrest - adjustable in height. It is used as a comfortable support of forearms. It also facilitates getting in and out of the wheelchair.
3. Footrest - The removable footrest is a safe and comfortable support for legs. It is adjustable in height.
4. Cross (under the seat) - allows the wheelchair to be easily folded for transportation and storage.
5. Brake - Pushing brake levers forwards engages them and protects the wheelchair against unforeseen movement and rolling down a slope.
6. Footplate - User's feet rest on it. It is possible to lift up the footrest, which can ease getting in and out of the wheelchair.
7. Backrest - made of smooth incombustible material, allows easy upholstery cleaning.
8. Seat - Made in similar way to the backrest with a seat cushion.
9. Main wheels - Majority of the user's weight rests on these 24 inches PU puncture proof wheels.
10. Push rim - Push rims attached to the main wheels allow self propelled driving of the wheelchair.
11. Drum brakes and brake handles (optional) - Allow controlling the wheelchair by an attendant, The brake handle is also equipped with parking brake function.
12. Anti-tip (optional) - Prevents wheelchair from falling backwards.
13. Quick-release axle - With this function wheels can be easily taken off, which facilitates the transportation of the wheelchair.
14. Front wheels - 8 inches turning wheels are installed on rotating forks, which makes maneuvering of the wheelchair possible.

Assembly description

View the Quick Guide using the following link:

mobilex.dk/medias/2022-01/qg_dolphin_barracuda_9484.pdf

Function description

Detaching, turning away, and adjusting the footrests in length

Standard footrests can be disassembled or pushed away on side.

To rotate and detach the footrest, first unblock it by pushing the safety-lock lever **1** forward, then turn the footrest outside and pull it upwards **2**. Attaching the footrest should be performed in reverse order by inserting it in base of frame and turning it inwards. The safety-lock will block automatically.

The wheelchair is equipped with height-adjustable footrests. To adjust their length one should release the locking clip **3**, adjust the length of the footrest by sliding the lower tube up or down, and then lock the clip.

While using the wheelchair outside, the footrests must be placed on height of minimum 4-5 cm from the floor. In addition, the footrests possess the ability of turning the footplates upwards **4**, which facilitates getting in and out of the wheelchair and disassembling the footrests.

To adjust the angle of the footplate first loosen the screw holding the footplate on the footrest, adjust the angle and tighten the screw.

Angle adjustable footrests (accessory)

The angle of these optional footrests can also be adjusted. To do it, press and hold the red button, move the footrest to a correct angle and release the button. The calf plate's height can be adjusted by unlocking the clip, moving it to the desired setting and locking the clip.

WARNING! You should never climb onto the footrests, it can cause the wheelchair to fall over. You should never lift the wheelchair by grabbing the footrests.

Armrests

The armrests can be adjusted in height, as well as be tilted backwards, behind the backrest, to facilitate the transfer of a passenger from or onto the wheelchair.

To adjust the height of the armrest, press and hold the red button **1**, adjust the armrest to the desired height and release the button.

To tilt the armrest backwards, one should press the lever **2** and lift the armrest up. To reattach the armrest, simply push it back down until the safety-lock **2** blocks itself automatically.

WARNING! It is prohibited to use the armrests as wheelchair carrying handles.


Parking Brakes

The wheelchair is equipped with parking brakes on both rear wheels to prevent rolling away.

To apply the brake push the lever forward **1**. To release the brake pull the lever toward the wheel **2**.

Always apply the brakes on both sides of the wheelchair.

WARNING! The parking brakes are not designed as driving brakes and only serve to lock the wheelchair (park position). They may not be used for braking while driving!

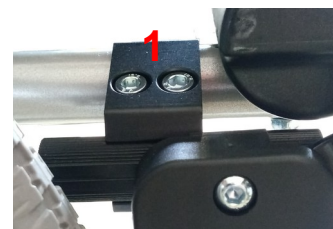The brakes can be adjusted for different types of drive wheels or seat settings.

To set the correct distance between brake and tire, first tilt the armrest backwards, then use a hexagonal wrench to loosen the 2 screws **1** that secure the brake on the topside of the guide bar. Slide the brake backwards or forwards to the correct position (approximately 10mm between the tire and the released brake shaft) and tighten the 2 screws. Reattach the armrest.

Drum Brakes (accessory)

The attendant brake levers are optional and may only be installed by the manufacturer or qualified service personnel. They assist the attendant in maneuvering the wheelchair user while driving. The set consists of two drum brakes, two brake levers which control them and two steel cables which connect the levers to the drum brakes. Brake levers are mounted under the push handles of the wheelchair.

To slow down the wheelchair, pull both brake levers up evenly to control the braking force. For full braking, pull the levers all the way up. To disengage the brakes, simply release the brake levers. You can use the brakes to maneuver the wheelchair by activating the brake on the side you want to turn to.

The drum brake handles also have a parking brake function. To engage the parking brake function, pull up the red element under the brake handle, and push it back down to release the parking brake.

NOTE: The brake levers at the push handles may only be operated by persons assisting the user, standing behind the wheelchair. These levers are not intended to be operated by persons sitting in the wheelchair.

NOTE: Never leave the wheelchair unattended without applying parking brakes.

NOTE: Always apply the parking brakes when the wheelchair is not in use.

24" rear wheels

The wheelchair is equipped with 24" main wheels with puncture-resistant polyurethane tyres. The wheelchair is delivered with the standard push rims. If the wheelchair user's grip strength is weakened, it is possible to change it to a push rim made of another material or add a cover to their push rim. It is also possible to change the distance between the push rim and the wheel.

One should be careful to not allow fingers to get between spokes during driving in narrow passages. To avoid such risk it is recommended to use a spoke guards.

The wheels on "Dolphin" and "Barracuda" chair can additionally be easily removed and fitted in by pressing the quick-release button located in the center of the wheel **1**.

To install the wheel place the end of the wheel axle back into the socket **2**, press and hold the button **1**, push the axle as far as it will go into the socket and release the button. Check if the wheel is fastened by pulling it outwards several times.

The wheelchair balance of "Dolphin" and "Barracuda" can be additionally adjusted by moving the wheel axle, which also requires to adjust the position of the turning wheels.

WARNING! We recommend such adjustment to be done by the qualified staff. An improper adjustment can decrease safety of usage of the wheelchair and decrease its durability.

Safety belts

There is possibility to attach hip safety belts. They are not included in the standard equipment.


Push handles with height adjustment

To adjust the height of the push handles one must loosen the lever **1** by turning it counter-clockwise. Then adjust the handlebar to the desired height **2** and tighten the lever **1** again by turning it clockwise. The lever's position can be adjusted separately by pulling it backwards and rotating it to avoid it sticking out to the sides.

WARNING! To ensure a secure push both handles must be set to the same height.

WARNING! The push handles have to be used only for pushing and steering the wheelchair. You should NOT use them for lifting or tilting the wheelchair!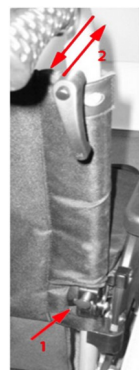


Backrest with angle adjustment (only on Barracuda)

To adjust the backrest angle press and hold the red knob **1**. Use the second hand to adjust the side of the backrest to desired angle **2** and release knob **1** to secure backrest in desired position. Repeat above steps on the other side of the backrest.

WARNING! Always ensure that the angle of both back tubes is adjusted to the same positions.

Tension-able Back (accessory)


The tension in this optional backrest can be adjusted with multiple straps. To adjust it first take off the Velcro cover off the backrest and adjust all straps one by one. A single strap is adjusted by releasing the bracket lock, pulling the Velcro apart and changing it's length by pulling it. Attach the Velcro so that the strap isn't loose, reattach the bracket lock and repeat for all the other straps. After adjusting all straps, attach the cover so that all Velcro straps on the back and the seat align.

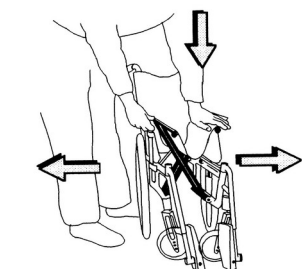Folding / unfolding the frame

To unfold the wheelchair, hold the seat tubes and push them apart outward, then push down on both of them (left image) until the seat upholstery is flat.

To fold the wheelchair, pull the seat upholstery up and guide one seat tube inwards (right image) until the wheelchair is folded.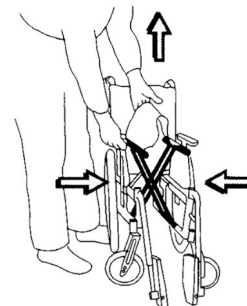

Step pedals

The step pedal on wheelchairs makes it easier for the attendant to avoid obstacles by tilting the wheelchair backwards (left image). The step pedal can be mounted onto the frame as showed on the picture (right image).

NOTE! DO NOT tilt the wheelchair with the handles, but use the step pedals!

Advanced settings and adjustments




WARNING! We recommend that all advanced adjustments are only carried out by qualified personnel. An improper adjustment may decrease safety when using the wheelchair and reduce the service life of the wheelchair.

WARNING! The correct seat depth should always be professionally determined by a therapist or specialist!

To adjust the seat depth, proceed as follows:

Loosen and remove the screws **1** on the upper and lower frame tubes, and slide the inner frame tube **2** forwards or backwards until you have reached the desired setting. Insert the screws **1** and tighten them. Repeat the procedure on the opposite side of the wheelchair.


Loosen and remove the screw **3** and adjust the holder to the same position as the chosen position for the screw **1**. After adjusting the seat depth it may be necessary to adjust the position of the brakes as well as the seat cushion. To adjust the cushion open it's cover, cut off the foam at the specified notches and close the cover again using a Velcro strap.

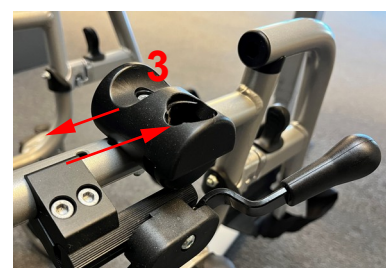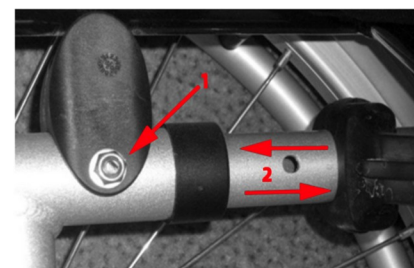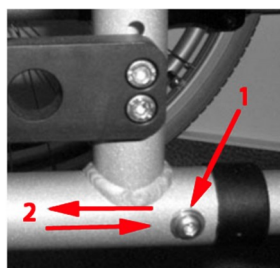Frame adjustments - changing the position of the rear and front wheels

The construction of the wheelchair makes it possible to change the height of the seat as well as allowing to shift the center of gravity of the wheelchair.

- Adjusting the seat height is done by changing the mounting position of the rear wheels (up/down) and changing the mounting position or size of the front wheels.
- Adjusting the balance of the wheelchair is done by changing the mounting position of the rear wheels (front/rear).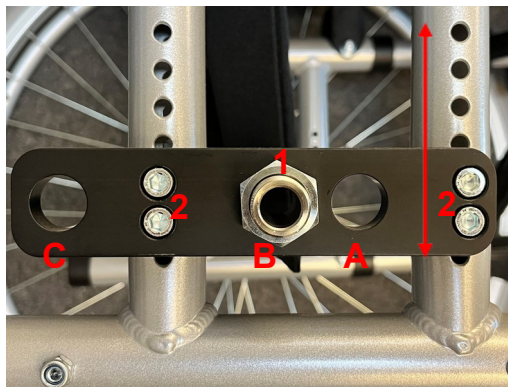

Adjusting wheel position (vertically and horizontally)

It is possible to adjust the rear wheels up or down, forwards or backwards to suit the tyre size and the personal preferences of the user.

If necessary, the seat height of the wheelchair can be adjusted by raising or lowering the axles of the rear wheels, which will require the position of the front wheels to also be adjusted.

The horizontal alignment (front / back) allows to influence the stability and manoeuvrability of the chair. The most common and neutral position is "B". If it is necessary to increase the manoeuvrability of the wheelchair, position "A" can be used. With the wheels in position "A" it is easier to manoeuvre the wheelchair but the stability to the rear is significantly reduced.

If the wheels are set to position "A" the chair should always be equipped with an activated anti-tip. Persons with amputated leg(s) should never use the wheelchair with the wheels in position "A" - the risk of tipping is too high. Leg amputees should set position "C" as it provides greater stability than the other positions. Even when using this setting, the wheelchair should be equipped with anti-tip accessories. As the manoeuvrability of the wheelchair is made more difficult with the axles in position "C", this setting is only recommended for leg amputees.

WARNING! The correct position of the wheels should only be determined by professional therapists or specialist companies.

There are 5 different height settings and 3 different horizontal settings possible.

To adjust the axles horizontally (front or back) proceed as follows:


1. Remove the wheels using the quick release axles.
2. Remove the wheel mounting bracket **1** and place it in the desired position (A, B, C).
3. Repeat the process with the opposite axle.
4. Re-insert the wheels using the quick release axles.

To adjust the axle height, proceed as follows:

1. Remove the wheels using the quick release axle.
2. Unscrew the wheel mounting plate by loosening the 4 screws **2**.
3. Move the plate up or down to your desired height dictated by screw slots.
4. Fix the axle mounting plate again with the 4 screws.
5. Repeat the procedure with the opposite mounting plate.
6. Re-insert the wheels using the quick release axles.

Transit wheels (accessory)

The optional 4" wheels can be mounted from the bottom of the wheelchair by screwing the two bolts of the accessory tube to rear slots of the axle mounting plate of a regular 24" wheel. The transit wheels have to be mounted from the outer side of the wheelchair.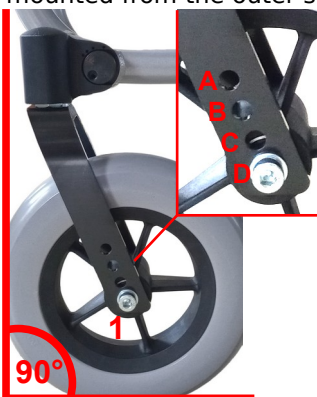

8" front wheels

The wheelchair is equipped with PU wheels 200x50 mm as standard. We recommend to stay with this type of tyres, but it is also possible to use pneumatic tyres. It is possible to install wheels with different sizes. This might be necessary after adjusting the rear axles. If the vertical position of the rear wheels is changed, it is always necessary to adjust the position of the front wheels accordingly. The vertical axis of rotation of the fork should always be set at a 90° angle to the ground (+/- 2°).

To make the adjustment, move the front wheel axle to another hole by loosening and removing the bolt **1**, moving the wheel up or down to the desired height and re-inserting and tightening it again in the selected hole. There are four holes available. In some cases it may be necessary to replace the standard front wheels (8") with a wheel of a different size (6" or 7").

Correct positions and front wheel sizes depending on desired seat height:

Desired seat height	45,0 cm	46,3 cm	47,7 cm	50,0 cm	51,3 cm
Axle mounting plate height setting (top to bottom)	1+2	2+3	3+4	4+5	5+6
Front wheel setting	B	B	C	C	D
Front wheel size	6"	7"	7"	8"	8"

Seat cushion



The wheelchair is as standard equipped with a 5 cm thick seat cushion.

- Always remove the seat cushion from the seat before folding the wheelchair frame.
- After unfolding the frame, put the cushion back on the seat.
- The zipper **2** allows you to easily remove the upholstery from the cushion in order to wash it.

Adjusting the wheelchair cushion (images on the right):

The wheelchair cushion can be shortened to fit the selected seat depth. To do this, open the back cover of the cushion and remove the inner fabric cap. There are several cuts on the back of the cushion to mark the different seat depths. Using a hobby knife or similar, cut the cushion completely through to the desired depth. Then replace the inner fabric cap and cover.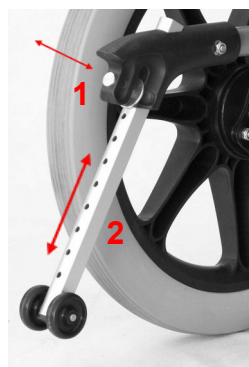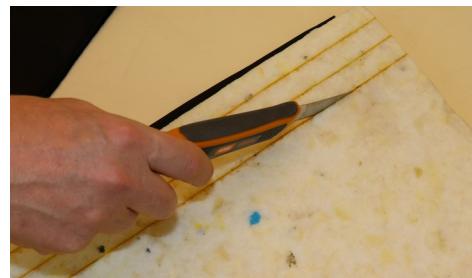

Anti-tip support (accessory)

This component increases rear stability and safety. It is especially recommended for wheelchairs of which center of gravity has been shifted rearwards. It should be installed on all wheelchairs used by leg amputees and is recommended for wheelchairs with the rear wheels mounted in the forward position.

To activate the support, pull and hold the locking pin backwards **1**, then move the support **2** to a desired position. Release the locking pin **1** so that it locks the support in this position.

To deactivate the support, simply move the support all the way up after pulling the pin and let go once adjusted.

The safe distance between the support and the ground should not exceed 4 cm.

WARNING! Proper installation of the anti-tip support should only be performed by qualified personnel.

Accessories

The device can be equipped with the following accessories (not a part of the standard equipment):

- ✓ Anti-tip
- ✓ Transfer wheels
- ✓ Safety belt
- ✓ Drum brakes
- ✓ Angle and height adjustable footrests
- ✓ Tensionable back (Velcro back)


4. Technical data

Dolphin (without / with drum brakes)	Art. 271340 / 271440	Art. 271344 / 271444	Art. 271348 / 271448	Art. 271351 / 271451	Art. 271354 / 271454	Art. 271357 / 271457
Seat-width	40 cm	44 cm	48 cm	51 cm	54 cm	57 cm
Seat-depth adjustable	42, 44, 46, 48 cm					
Seat-height min. - max.	45 - 52,5 cm; steps of 1,5 cm					
Total width	60,30 cm	64,30 cm	68,30 cm	71,30 cm	74,30 cm	77,30 cm
Weight	15,60 / 16,10 kg	15,90 / 16,40 kg	16,20 / 16,60 kg	16,75 / 17,25 kg	19,40 / 20,10 kg	22,64 / 23,14 kg
Tyres front/back (models with drum brakes only)	PU/PU	PU/PU	PU/PU	PU/PU	PU/PU	PU/PU
Max. user weight	150 kg				200 kg	

The standard version of the Dolphin wheelchair has passed crash tests according to ISO 7176/19. The heavy version (200kg) was not tested. The standard version can be used as a car seat for users weighing 75kg and 136kg. The wheelchair is designed to be used forward-facing when used as a seat in a motor vehicle. Whenever possible, the user should use the car seat and seat belt! Only use fastening systems that comply with the ISO 10542 standard to secure the wheelchair to the vehicle floor.

Barracuda (without / with drum brakes)	Art. 271840 / 272040	Art. 271844 / 272044	Art. 271848 / 272048	Art. 271851 / 272051
Seat-width	40 cm	44 cm	48 cm	51 cm
Seat-depth adjustable	42, 44, 46, 48 cm			
Seat-height min. - max.	45 - 52,5 cm; steps of 1,5 cm			
Total width	60,30 cm	64,30 cm	68,30 cm	71,30 cm
Weight	20,2 / 20,7 kg	20,5 / 21kg	20,8 / 21,3 kg	21,35/ 21,85 kg
Backrest inclination	0 - 30°			
Max. user weight	120 kg			

Product label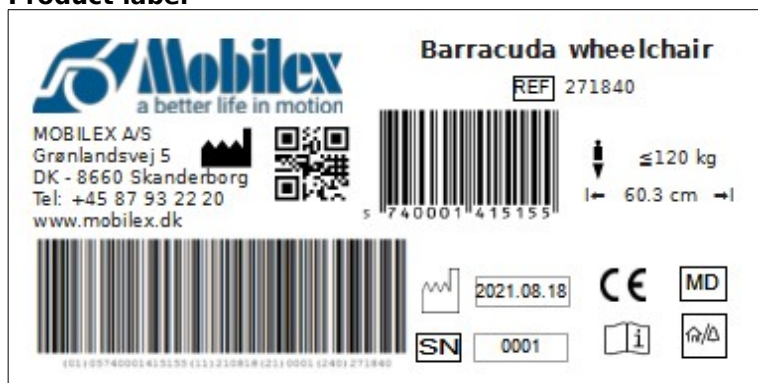

Example of the product label

	QR-code link to homepage		Manufacturer
	CE marked product		Item Number
	Observe the user manual		Max load
	Medical Device		Manufacturing date
	To be used indoor and outdoor		Serial no.
	Max width		

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. For disinfection, use only disinfectants certified for use with medical devices available locally.

Wheels can be cleaned with hot water with a mild cleaning agent. After cleaning the wheel bearings should be lubricated with oil periodically or when the wheels start squeaking.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components


Recommended service life: Normally 5 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

It is recommended to transport the device folded, in a car trunk. If that method of transportation is unavailable, the folded wheelchair can be transported behind the front seats of the car (right image).

WARNING! The wheelchair is not the car seat. During traveling in vehicles sitting on the wheelchair is not allowed. The chair should be folded and safely immobilized.

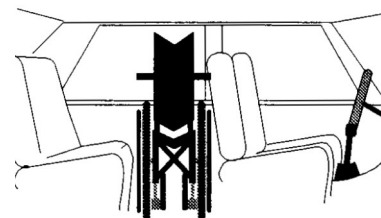Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

Parts, which should be regularly inspected

Part name	Control type	Control frequency
Spokes	Loose spokes can cause deformation of the rear wheels. Contact your point of sale or local bicycle service in case of loose spokes.	If the problem occurs
Wheel axles	Remove the hair or accumulated dirt	If necessary
Push Rims	Excessively scratched push rims ought to be exchanged since they can wound the user's hands while driving the wheelchair	If necessary

Part name	Control type	Control frequency
Brakes	The braking force depends on the tyre pressure. The efficiency of brake operation can be also affected with dirt accumulated on tyres. Keep the brakes clean by wiping them with a damp cloth to remove dirt, and also lubricating the funnel of screws, on which the brake levers turn.	Assure correct brake operation at least once a month
Frame	Maintain the wheelchair in the highest cleanness for better comfort of the user.	At least once a month
Turning wheels	The area between the fork and the front wheel should be kept clean, because of dirt accumulating there, which can cause faster wear of turning wheel bearings. To clean the front wheel, one should disassemble it by unscrewing it from the fork, remove all dirt, and then preserve the metal elements of the wheel (i.e. technical grease).	The maintenance is necessary once a month or more often depending on the environment of use.
Detachable elements	Check the condition of detachable elements of the wheelchair, if screws are loose they should be tightened.	In case of intensive exploitation of the wheelchair the inspection should be carried out once a month.
Quick-release axles of the rear wheels	One should clean and then lightly lubricate the quick-release axles of the rear wheels, with a damp cloth the slots of fixing the quick-release axles need to be cleaned from all dirt. The wheel should be laid on the push rim (never opposite way, because the quick-release axle can be dirty, which can cause its seizing in the main axle opening. In case of solid particles occurring on the surface of the quick-release axle of the wheel, use is not allowed.	Lubricating should be carried out periodically.

6. Most common problems and solutions

Problems	Solutions
The wheelchair seems to tilt to one side	<ul style="list-style-type: none"> • check fastening of the front turning wheels • check the displacement of the wheelchair's center of gravity • one of the rear tires might be inflated more than the other one
The wheelchair is hard to push	<ul style="list-style-type: none"> • check fastening of the front turning wheels • remove dirt or entangled hair from the front turning wheels • too large load applied on the front turning wheels: move the center of gravity
The wheelchair is hard to turn	<ul style="list-style-type: none"> • ensure, that bolts fastening the front turning wheels are not tightened too strong • remove dirt or entangled hair from the front turning wheels
Brakes are not working correctly	<ul style="list-style-type: none"> • adjust the distance between the brake and tires
It is hard to fold and unfold the wheelchair	<ul style="list-style-type: none"> • the upholstery fitted too tight • clean up or apply grease to the cross under the seat
The wheelchair is not stable	<ul style="list-style-type: none"> • ensure, that all screws/bolts and nuts are tightened • ensure, the wheelchair is properly adapted to the passenger

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

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