Rollator ELEPHANT AIR
Item no. 317001 & 317002

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione

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1. Introduction

Thank you for buying a high-quality Mobilex product. These user instructions contain a detailed product description and important guidelines for correct use of the product. You must read the instructions carefully before using the product. Please always observe all guidelines, in particular the security instructions, in order to use the product safely for a long time.

Intended use of the device

The Mobilex product is medical device which has been designed to increase the mobility of persons who have difficulties standing or walking. The rollator enables those persons to regain the ability to perform daily chores. The rollator is suitable for persons who as a result of various diseases need stable walking support. The rollator should only be used as a support for standing, walking and sitting. Any other use is prohibited.

Indications

The rollator is a medical device designed to help people with functional limitations. The indications for use are diseases that cause mobility limitations and disabilities, obstructing independent sitting, standing and walking. Therefore, there is a need for an additional stable support when performing the above activity. Indications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status by professional physician or physical therapist, but main target user groups are elderly people, persons with paresis, rheumatic diseases, and other people with limited mobility of the lumbar spine, knees or hips. Using the product for any purpose other than those described above is prohibited.

Contraindications

Rollators are not indicated for those persons who can't maintain proper body balance in a standing position. The device must not be used if the user has problems with impaired perception. Contraindications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

If you have any questions, please contact your dealer or Mobilex A/S

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
--	--

2. Security instructions

- Use only the product if it is technically flawless
- In case of any product damage or faults please contact your dealer immediately
- Observe the data on the product label
- Use the product only for the described purpose
- Please notice that there is a risk of injury at all joints on the rollator
- Always check before use that the wheel brakes function perfectly
- Check tyre pressure regularly, as it should always be 2.1-2.4 bar (30-35 psi)
- Regularly check and retighten the finger screws
- Use the rollator only when it has been fully unfolded and the locking device has been activated
- The rollator should only be used on a flat stable surface, indoors or outdoors
- The seat may only be used when the brakes have been firmly locked
- Both brakes should always be activated or locked simultaneously
- Max. load of the rollator is 200 kg in total
- Max. load of the shopping bag is 5 kg
- It is not allowed to carry an extra person on the seat
- The product must not be changed without the manufacturer's written permission
- The rollator must always be braked when not in use
- Remember to retighten the finger screws regularly

3. Description

The Elephant Air rollator has a powder coated reinforced frame with 4 wheels. The light-weight frame makes it easy to fold it together and offers easy storage and transport. The rollator runs easily and is sufficiently stable. The height of the underarm supports can be adjusted. The rollator is delivered with a mounted seat, backbelt, crutch holder and shopping bag.

Parts supplied and mounting instructions

The rollator is delivered in a carton with the following contents:

- ✓ This user manual
- ✓ 1 rollator including a mounted seat, backbelt, crutch holder and shopping bag

Control upon delivery

After unpacking the rollator please check if anything is missing, if all parts shown in the drawing have been delivered. If any parts are missing, please contact your dealer immediately.

Unfolding the rollator

- Place the rollator with the wheels on the floor. Grasp the black lock and tilt it up and out of the tubes at the seat, the rollator unfolds. Then push the two tubes downwards until you hear a click in the locking mechanism. The rollator is now unfolded and secured against collapsing.
 - To be quite sure that the frame has been fully unfolded try to press the rollator together
 - Check that the levers for height adjustment have been firmly tightened
 - Check that both handles are at the same height and stable
- Use of a defective rollator is strictly forbidden*

Always check before use the function of all parts of the rollator. If the above recommendations are not followed, injuries may occur.

Folding the rollator

- Pull the strap at the seat until the frame folds together
- Now the rollator is folded together

Height adjustment of push handles

- Press the adjustment button.
 - Adjust the height
 - Make sure that the adjustment button pops out again.
 - Make sure both push handles are adjusted in the same height.
- Both handles should always be at the same height.*

Adjust the height of the push handles so that the handles are at wrist height when the user stands next to the rollator with arms vertically downwards. This means that the arms have an angle of 20°- 30° when using the rollator.

Using the hand brakes

- To brake, pull both brake levers up.
- The harder the brake handles are applied, the stronger the braking effect.
- To STOP, keep the brake handles applied.
- To MOVE the rollator, release the brakes again. They will be released automatically.

*Always engage the parking brakes when you are not using the rollator!
 Never release the parking brakes when you are sitting on the rollator!
 If you park the rollator on uneven or sloping paths, the parking brakes must always be activated!*

Locking / releasing brakes

- To bring the rollator to a standstill, pull the brake levers upwards.
- To park, push the brake levers down until you hear a "click" sound.
- The rollator may be used for sitting only when the brakes are locked.
- Before driving off again, stand in the direction of travel and release the brake levers by pulling them upwards.

*When parking, always lock the brakes.
 Never use the seat without the brakes locked!
 Never lean backwards when sitting. It is recommended to use only the front seat area and not to sit on the bag or lean backwards.*

Accessories

No accessories available for the rollator.

Transport recommendations

It is recommended to transport the rollator in the folded position. Its construction allows easy unfolding and folding. You should check that the rollator is in a stable position during transport to avoid any deformation.


Use: Stand in the middle behind the back wheels. Grab the push-handles and smoothly start walking. Due to the special design of the brakes, only a minor effort is required to use the brakes.

4. Technical data

Rollator Elephant Air	Creme Royal 317001	Matte black 317002
Height of the push handles (adjustable)	87,3 - 104,8 cm	
Recommended user heights	150 - 195 cm	
Seat height	62 cm	
Total width opened / folded	67 cm / 28,0 cm	
Inner width between push handles	45,5 - 40,7 cm	
Length (depth)	80,5 cm	
Weight (+ bag 380 g)	10,68 kg	
Max. load (rollator / bag)	150 kg / 5 kg	
Brakes	2 lockable with parking function	
Wheel sizes	Front and back: 12" / 291 mm	

Product label

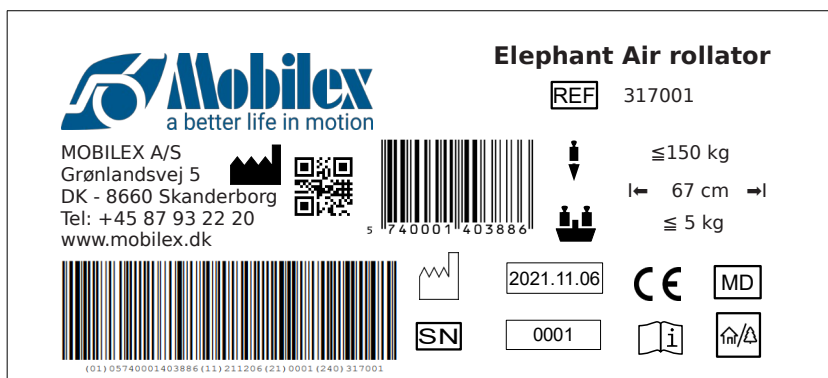The product label is placed on the lower frame

Example of product label Elephant Air 317001

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device
	Only to be used indoor		

Overview icon meaning

5. Maintenance and care

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General retightening of the product and checking of tyre pressures

Recommended service life: Normally 10 years depending on use.

Cleaning the wheels

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

Warning

If the product is exposed to very calcareous water, a binding and accumulation of lime on wheel bearings and other moving parts may occur. In this case, the product can be cleaned with a limescale remover and then lubricated with lubricating oil.

Warning

Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

Storage

We recommend storage in a dry place.

Transport recommendations

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

6. Troubleshooting

<i>Problems</i>	<i>Possible Causes</i>	<i>Solutions</i>
The rollator is difficult to guide or push	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
The brakes do not work or work only in one side	The brake shoes are worn, or the brakes are not adjusted correctly	Adjust the brakes. Replace the brake shoes if needed (→ service centre)
The seat does not fit in when the rollator is unfolded	The locking device is in a wrong position or defective	Check that the locking device is in a correct position. Replace it if needed (→ service centre)
The rollator runs crooked	There is not enough air in the tyres	Check tyre pressure

7. Disposal of the product

The product may not be disposed of with household waste but must be brought to the local recycling site.

8. Guarantee

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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ENGLISH

Declaration of Conformity

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product are classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

Intended purpose

Indicated for people with mobility limitations who have problems with walking, standing or sitting without stable support

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