

Mobilex PU-fitting device item no. 211501 User manual



**MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
Denmark
www.mobilex.dk**

1. General information

Dear customer,

This user manual contains a description of the product and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these. We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products. By questions please take contact to your dealer or directly to Mobilex A/S.

Mobilex A/S
Grønlandsvej 5
DK-8660 Skanderborg
Denmark

Phone: +45 87 93 22 20
Telefax: +45 87 93 17 77
E-Mail: info@mobilex.dk


2. Safety requirements

- Make sure that this user manual is read by all persons using the device. Manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments which are possible on the device the user must take care not to squeeze the fingers or other body parts between two moving elements of the device
- The device must only be used on a stable surface.

3. Product Description

The Mobilex PU-fitting device 211501 was developed as an assembly aid for the installation of polyurethane "rubber" tires with a size of 20" to 28". It facilitates and accelerates the installation of polyurethane tires and thus can significantly contribute to a significant time saving during tire mounting.

Scope of delivery

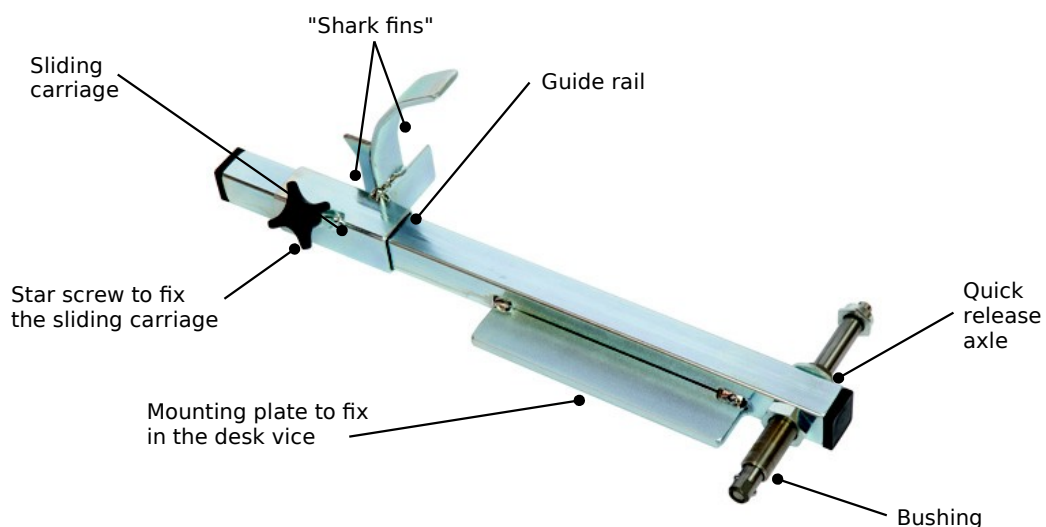


Assembly

The PU-fitting device 211501 is supplied already assembled

Use



1. Attach the tire changer in the vice and attach the wheel with the quick-release axle to the bushing on the PU fitting device.

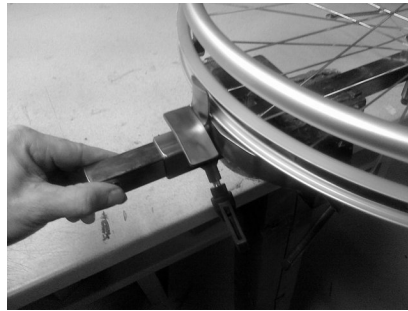

2. Move the sliding carriage on the PU fitting device so that the vertical "shark fin" easily applies to the rim and fix the sliding carriage with the star screw.



3. Put the PU tire on the wheel and push the tire as far as possible into the rim. Make sure that the vertical fin is located between tire and rim.



4. Turn the wheel by hand once around to the point from which you started. The tire should be mounted now.



5. Now remove the wheel from the PU fitting device by pulling it up and detaching it from the fin. Note that this can be a bit stiff.

4. Cleaning & Maintenance

Normal dirt on metal and plastic parts can be removed with standard cleaning agents. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (no solvents or abrasives).

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional

Device should be stored in dry places with temperature always above 0 degrees. The device should not be stored in places exposed to extreme weather conditions.

5. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

6. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.