SWIVEL BATHER

Art. 302022

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

The swivel bather is a medical device intended only to be used by patients that because of invalidity need support during entering or leaving the bathtub and during washing. The swivel seat makes it possible for the patients to pass elevated side walls of a bathtub with or without the assistance of a helper. Physically handicapped persons can use it as stable support and it gives an easy access to a bathtub and also all human body parts during washing. The device should be used inside bathrooms only.

Indications

The device is indicated for people with functional limitations, people in wheelchairs and other people with limited mobility of the lumbar spine, knees or hips, and for all those who have difficulties performing everyday personal hygiene in a bathtub. Any use other than stated above is prohibited.

Contraindications

The device is not indicated for those patients who can't maintain proper body balance in a sitting and standing position. The device must not be used if the user has impaired perception. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- Use caution when assembling or adjusting any parts of the device.
- We recommend to use help of an assistant for getting in and out of the swivel bather.
- The swivel bather must only be used in traditional bathtubs with sides of equal height. Make sure that the locking bolts are correctly adjusted before use and that the 4 arms lay flat and stable on the sides.
- Use swivel bather only for seating. Do not stand on the seat.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Base frame with 4 arms;
- ✓ 1 PU seat;
- ✓ 1 PU backrest;
- ✓ 2 Backrest tubes;
- ✓ 2 Armrests;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

The swivel bather consists of a base frame made of aluminum, a soft PU seat and a PU backrest. The four arms of the base frame are covered with protective rubber caps/supports to be laid down on top of the side walls of the bathtub. The device is equipped with a mechanism that allows the seat to turn. This function makes entering or leaving a bathtub even easier.

The device consists of the following elements:

1. PU backrest
2. Detachable armrest
3. PU seat
4. Threaded rod for chair fixation
5. Rubber support (for bath)
6. Swivel lock lever (under the seat)

Assembly description

Slide the backrest tube ends into the holes at the back of the main frame and lock them with clips. Attach the PU backrest to the tubes. Slide the armrests tube ends into the holes at the sides of main frame, by pushing the button placed on the tube on both sides.

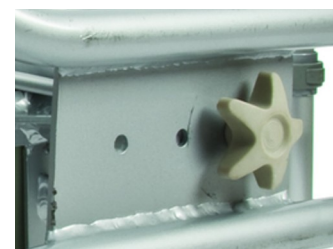
Function description

Positioning the device

Put the bath seat on the bathtub. Check if all 4 arms lay flat and stably on the sides of the bathtub. Fix the main frame setup with the 2 threaded rods located under the frame so that they press against the sides of the bathtub.

Adjusting the position of the seat

It is possible to adjust the position of the seat (to the left or right) by moving it's assembly into one of the other adjustment holes located on the central fixing plate (3 positions available). This is done by unscrewing the plastic knob placed underneath the seat. Move the seat to a new position, fit and tighten the plastic knob again. Check if the seat can rotate freely when the locking lever is lifted.

Using the chair after installation

Turn the seat towards the bath edge by pulling the locking lever up and rotating the seat frame. Turn yourself backwards to the seat, place your hands on the armrests and sit on the swivel bather. Pull the locking lever up and rotate yourself with the seat frame towards the center of the bathtub. Make sure that the locking lever is locked in that position, so the seat won't move unintentionally. To get out of the bathtub repeat the steps above,


Accessories

There are no accessories provided for this device.

4. Technical data

Sliding bath bench	Art. 302022
Overall width	71 cm
Overall length	54 cm
Overall height	57 cm
Seat width x depth	41 x 34 cm
Width between armrests	56 cm (at the top) / 51 cm (at the bottom)
Height of the armrests	38 cm
Weight of the product	4,88 kg
Max. user weight	130 kg
For bathtub widths (internal)	57-66 cm

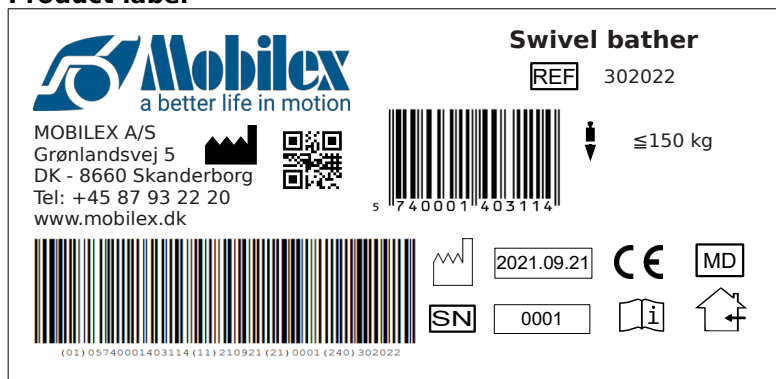Product label

Example of the product label

	QR-code link to homepage		CE marked product
	Manufacturer		Manufacturing date
	Observe the user manual		Max load
	Serial no.		Medical Device
	Item Number		Only to be used indoor

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. The entire product can be disinfected with a commercially available disinfectant. Always follow the disinfectant manufacturer's instructions for use and safety when disinfecting.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

Problems	Possible causes	Solutions
Soft seat pad is not fixed in position (shaking or moving)	One or more of seat holding pins are not inserted correctly in to the frame holes	Take the bath seat out of the bathtub and turn it upside down to check if seat holding pins are correctly inserted. If necessary push the seat pad by pressing all pins into their holes on the main frame
The bath seat doesn't lay flat and stable on the sides of the tub	Rubber arm supports are not in the same parallel and flat position	Check if all 4 rubber supports are placed in a way, so that their flat sides are on the bottom touching the bathtub. Adjust the position of rubber supports if necessary

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

mobilex.dk/medias/2021-07/ce_aqua_drejbar_badekarsaede_aluminium_302022_8307.pdf

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