## Mobilex "Double Folded" DF ramps

Art. DF-180, DF-240 & DF-300

**Montage- und Gebrauchsanleitung**  
**Montage- og brugervejledning**  
**User Manual**  
**Manuale d'uso e manutenzione**

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## 1. Introduction

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

### Intended use of the device

The Mobilex DF Ramps are medical devices which is intended to be used only to assist patients in wheelchairs or scooters. They enable the passage of elevated points (e.g. Stairs) with or without the assistance of a helper. Mobilex Ramps thereby enables patients, sitting in a wheelchair because of invalidity, to enter or leave buildings or cars, buses etc. The ramps can be used as well inside as outside. Any other use is prohibited

By questions please take contact to your dealer or directly to Mobilex A/S.

<b>Mobilex A/S</b> Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 E-Mail: info@mobilex.dk
--	---

## 2. Safety requirements

- Make sure that this user manual is read by all persons using the device. Manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments which are possible on the device the user must take care not to squeeze the fingers or other body parts between two moving elements of the device
- The device must only be used on a stable surface.
- Ramps must not be loaded with more than the "max load" described in Technical Data and on the label.
- The ramps can be used at maximum inclination of:
  - DF-180            36 cm**
  - DF-240            48 cm**
  - DF-300            60 cm**
- Be careful during folding or unfolding of the ramp. There is a risk of finger stuck or cut between parts of the ramp.

## 3. Description

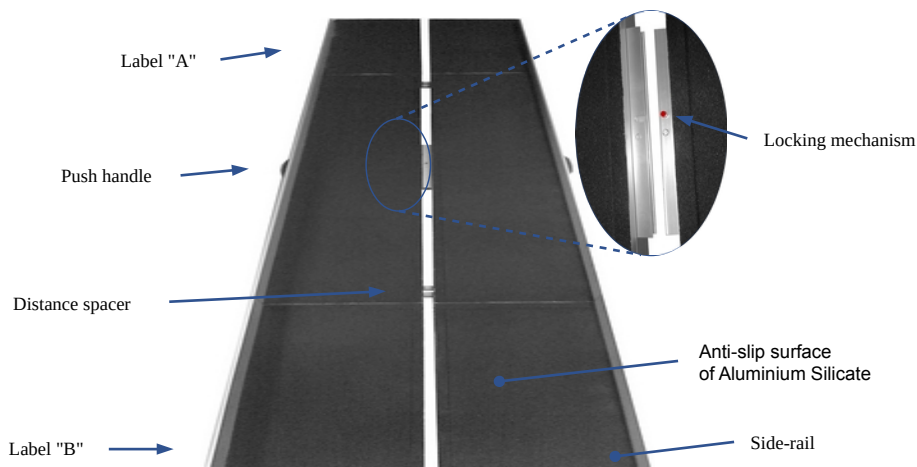
### Product description

Mobilex DF-Ramps are foldable platforms made of aluminium. The surface is covered with special material for anti-slipping purposes. To make handling and transportation easier ramps are divided in to two foldable halves than can be assembled together for usage.

SF-Ramps are supplied in cartons. The contents are:

- ✓ 2 Mobilex DF ramps, left and right side
- ✓ 1 user manual

### Set up & Function description

- Ramps are marked with label **A** and label **B**.
- Label **A** is placed on the upper side of the ramp (the side placed on the hindrance).
- Label **B** is placed on the lower side of the ramp (the side placed at the ground).
- The ramp parts must be fitted together with the red springs.
- The ramp must be placed so that the high side protection is outwards.
- The ramp must be placed on a stable surface.
- Check that the ramp is stable & ramp halves are connected before driving on it.

### Transport recommendations

Alligator ramps type DF can be disassembled in to two halves and folded (with hinge) for transport. Always assure that ramp during transportation is safely secured against unintended movements.

### Accessories

There are no accessories provided for Alligator Ramps.

### 4. Technical data

Mobilex DF-Ramps	DF-180	DF-240	DF-300
Length	180 cm	240 cm	300 cm
Width (per rail)	79 cm	79 cm	79 cm
Surface width	75 cm	75 cm	75 cm
Height of side protect.	6 cm (inner)	6 cm (inner)	6 cm (inner)
Max. Elevation / height	12° / 36 cm	12° / 48 cm	12° / 60 cm
Weight (1 pair)	16 kg	22 kg	27,2 kg
Max. load	300 kg	300 kg	300 kg
Folded size cm (LxWxH)	91 x 36 x 19	124 x 40 x 19	154 x 40 x 19

### Product label

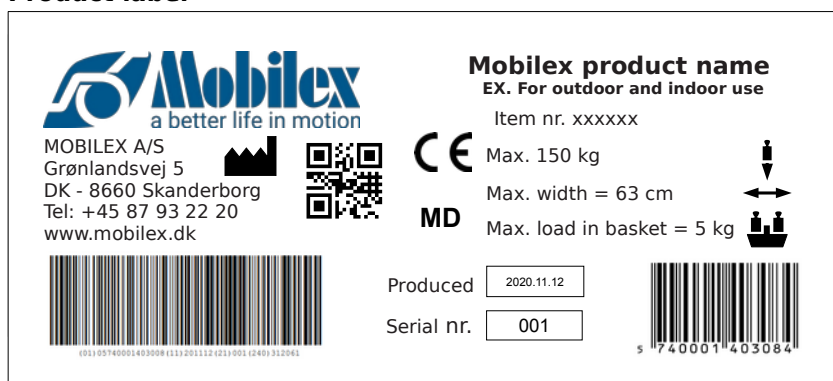
Image of the product label (not original)

## 5. Maintenance and cleaning

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional

- Lighter dirt and dust can be removed with a wet cloth.
- Heavier types of dirt can be removed with a non-corroding, mild washing agent.
- You can also use garden hose to clean the ramp.

**Recommended service life:** Normally 10 years depending on use.

### Warning

 Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

## 6. Reuse

The product is suitable for recycling. Before recycling, where a new user is to take over the product, the product must be checked according to point 5.

## 7. Most common problems and solutions

<i>Symptoms</i>	<i>possible cause</i>	<i>what to do?</i>
Ramp is not stable	The surface under the ramp is not flat	Check the position of the ramp. Put it on flat, smooth surface
Ramp is not stable	Ramp is putted on the surface upside down	Check if the A label is in the upper end and B label in the lower end of the ramp. Correct position if needed.
Ramp doesn't open	The lock strap is not loosen	Loosen the lock strap

## 8. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

## 9. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,