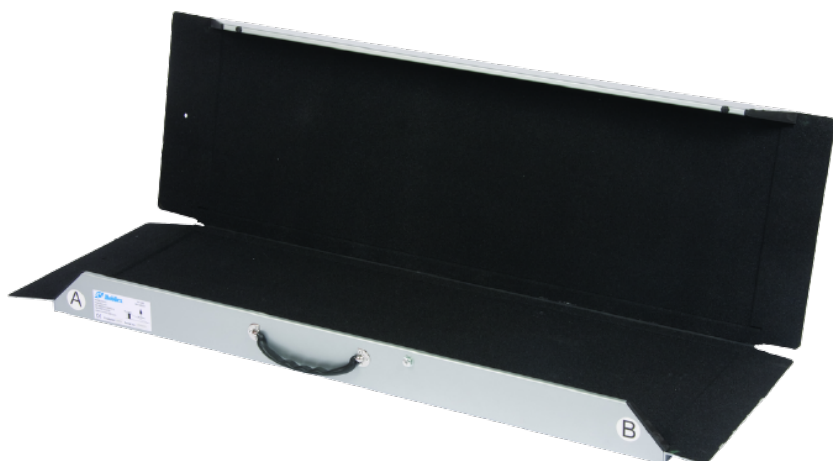
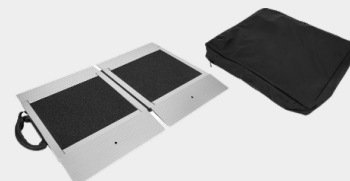
**SF RAMPS**  
**"SINGLE FOLDED"**  
Art. SF-040, SF-090, SF-120 & SF-150

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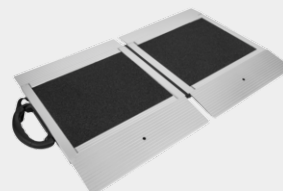
SF-ramps of aluminium with a friction surface and foldable in the longitudinal direction.  
SF flat ramp offers the opportunity for helpers to stand on the ramp together with the wheelchair and the user.

SF-040 ramp in use with bag that can be attached to the wheelchair

SF-040 and SF-090 ramps with a carrying handle and hinged at the middle.

SF-120 and SF-150 in use. Foldable, hinged and portable at the handles like a suitcase.

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## 1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions for use contain a description and important information about the use of the medical product. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the product. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

### Intended use of the product

To be used to assist people in wheelchairs or scooters enable the passage of elevated points.

### Indications

Ramps are indicated for people with mobility limitations who due to problems with walking, standing or sitting are forced to use mobility devices such as wheelchairs, scooters, rollators etc. Ramps are indicated for elimination of architectural obstacles by smooth connection of the level differences that a wheelchair user has to overcome and thus they allow the wheelchair user to reach places normally inaccessible to him/her. The ramp is intended for use independently or with the help of an attendant. Using the ramp for purposes other than those indicated above, is prohibited.

### Contraindications

Contraindications cannot be tied to a specific clinical picture. The choice of a suitable ramp type and length must always be made with the help of a specialist doctor or physiotherapist. Observe the safety information in chapter 2.

### Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

### Medical incidents and unwanted events

In case of any unwanted events related to the product, please contact the dealer or manufacturer:

Mobilex A/S	Phone:	+45 87 93 22 20
Grønlandsvej 5	Telefax:	+45 87 93 17 77
DK-8660 Skanderborg	Email:	info@mobilex.dk
Denmark		

## 2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments which are possible on the device the user must take care not to squeeze the fingers or other body parts between two moving elements of the device
- The device must only be used on a stable surface.
- Ramps must not be loaded with more than the "max load" described in Technical Data and on the label.
- The ramps can be used to overcome obstacles at maximum heights of:

<b>SF-040</b>	<b>8 cm</b>
<b>SF-090</b>	<b>18 cm</b>
<b>SF-120</b>	<b>24 cm</b>
<b>SF-150</b>	<b>30 cm</b>
- Be careful during folding or unfolding of the ramp. There is a risk of finger stuck or cut between parts of the ramp.

### Parts supplied

Each product package should contain: 1 ramp, 1 user manual.

### Preliminary check

Check the contents of the box against the above list. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

## 3. Description

### Product description

SF-Ramps are foldable platforms made of aluminium. The surface is covered with a special anti-slipping material.

**The ramp consists of:**

1. Top support
2. Label A
3. Product label
4. Carrying handle
5. Side protection
6. Anti-slip surface of aluminium silicate
7. Label B

**Function description**

**Placing the ramp**

Place the side labeled "A" on the elevated surface, and the side labeled "B" on the ground level. Make sure the ramp is steady and placed on a stable surface before use.

**Assembly**

This device requires no assembly instructions.

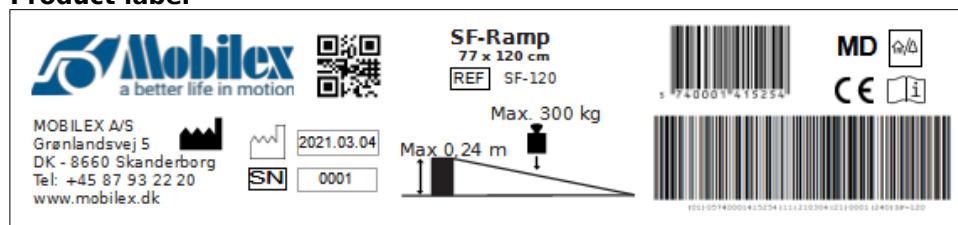
**Accessories**

There are no accessories provided for this product.

**4. Technical Data**

SF Ramps	SF-040	SF-090	SF-120	SF-150
Length	40 cm	90 cm	120 cm	150 cm
Width	71 cm	77 cm	77 cm	77 cm
Surface width	71 cm	73 cm	73 cm	73 cm
Height of side protect.	--	6 cm	6 cm	6 cm
Max. Elevation / height	12° / 8 cm	12° / 18 cm	12° / 24 cm	12° / 30 cm
Weight	4 kg	8 kg	10,5 kg	12,5 kg
Max. load	300 kg	300 kg	300 kg	300 kg
Folded size cm (LxWxH)	45 x 37 x 5	90 x 40 x 10	120 x 40 x 10	150 x 40 x 10

**Product label**

Example of product label

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Batch no/ serial no
	CE-marked product		Item Number
	To be used indoor and outdoor		Medical Device
	Max Elevation / height		

Meaning of icons

**5. Care & Maintenance**

Normal dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The product basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary

**Expected service life:** Normally 10 years depending on use.

**Storage:** We recommend storage in a dry place at a temperature above 0, away from intense sunlight.

**Transportation**

The ramp can be locked in a folded position. It should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

## Reuse

The product is suitable for reuse. Before a new user takes over the product, it must be checked according to point 5.

## 6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
Ramp is not stable	The surface under the ramp is not flat	Check the position of the ramp. Put it on a flat, smooth surface
Ramp is not stable	Ramp is deployed upside down	Check if the ramp is positioned correctly
Ramp doesn't open	The lock strap is not loosened	Loosen the lock strap

## 7. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling center.

## 8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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**Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark, SRN: DK-MF-000021885**