PANTHER ROLLATOR

Art. 318003, 318004

**Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur
Gebruikershandleiding**

Panther lightweight rollator
has a champagne aluminium frame
For both indoor and outdoor use
Available in 2 sizes
Standard (62 cm)
Small (55 cm)
Tested for 150 kg

Height adjustable
Push handles.
No tools needed
Integrated tilt
pedal and 2
lockable brakes

Removable bag with
two handles that can also
be used for shopping

The rollator folds
easily sideways
Takes up very little
space when folded,
for easy transport

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mobilex.dk/medias/2024-06/ce_panther_318003_04_16406.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

This product is medical device which has been designed to increase the mobility of persons who have difficulties standing or walking. The rollator enables those persons to regain the ability to perform daily activities. The rollator is suitable for persons who as a result of various diseases need stable walking support. The rollator should only be used as a support for standing and walking. The rollator seat can be used for sitting down and resting. Any other use is prohibited.

Indications

The rollator is a medical aid that helps people with functional limitations. The indications for its use are illnesses that lead to mobility restrictions and disabilities that hinder independent standing and walking. An additional stable support is therefore required when carrying out these activities. The indications cannot be related to a specific medical condition and depend on the individual assessment of the potential user's state of health by a doctor or physiotherapist. However, the main target groups are older people, people with paresis, rheumatic diseases and other people with limited mobility of the lumbar spine, knees or hips. The use of the product for any purpose other than that described above is prohibited.

Contraindications

Rollators are not indicated for those persons who can't maintain proper body balance in a standing position. The device must not be used if the user has problems with perception. Contraindications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product is suitable for indoor and outdoor use and can be used by private individuals in domestic environments, in home care, in health and care facilities and in hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S	Phone:	+45 87 93 22 20
Grønlandsvej 5	Telefax:	+45 87 93 17 77
DK-8660 Skanderborg	Email:	info@mobilex.dk
Denmark		

Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device and the bag must not be loaded with more weight than indicated in Technical data.
- All wheels **MUST** be in contact with the floor at **ALL TIMES** during use. This will ensure that the rollator is properly balanced and should decrease the risk of accidents.
- Use caution when assembling or adjusting any parts of the device.
- Always check if the brakes function properly before use.
- Use the rollator only when it has been fully unfolded and the locking device has been activated.
- The seat may only be used when parking brakes have been activated.
- Both brakes should always be activated or locked simultaneously.
- It is not allowed to carry a person or object on the seat while the rollator is in motion.
- Parking brakes must always be engaged when the rollator is not in use.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Rollator folded incl. fitted seat, back belt, cane holder and shopping bag;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

The rollator has 4 wheels and an aluminum powder coated frame. Because of the materials used the entire rollator is corrosion resistant. The light-weight frame makes it easy to fold it together and offers easy storage and transport. The rollator runs easily and is sufficiently stable. The height of the push handles can be adjusted. The rollator is delivered with a mounted seat, a back strap, a cane holder and a shopping bag. The rollator is equipped with reflectors on the push handles, bag and front wheel forks.

The rollator consists of the following elements:

1. Push handle (Material: PP+TPR)
2. Brake lever (Material: PP)
3. Top cane holder part
4. Seat tube
5. Handle height adjustment holes with a locking pin
6. Bottom cane holder part
7. Brake
8. Rear wheel
9. Tip-pedal
10. Backrest (Material: PA)
11. Folding lock clip
12. Seat with strap for folding (Material: PA)
13. Shopping bag (Material: PA)
14. Front turning wheel
15. Reflectors

Assembly description

This device doesn't require an assembly instruction.

Function description

Folding/unfolding the frame

To unfold the frame unlock the folding lock clip and press down on the seat tubes until the frame unfolds and "clicks", which indicates the rollator is locked in an unfolded position. To fold the frame pull up the folding strap and finish folding the frame by pushing seat tubes towards each other and locking them together with the folding lock clip.

WARNING! Seat tubes must snap into place firmly on both sides when unfolding the rollator. All parts must be firmly locked before the rollator can be used. Failure to observe these warnings may result in injury.

Push handle height adjustment

To adjust the height of arm supports press the height adjustment pin and move the arm support one setting away. Repeat until the desired setting is reached and repeat the process for the second handle.

NOTE: Both handles should always be at the same height. The correct height of the push handles should always be on the height of the user's wrists when standing with arms along the body.

Brakes

The brakes can be gradually activated by pulling the brake levers up with increasing force. Brakes are fully engaged when brake levers touch the handles and the rollator is fully stopped. To stop braking let go of the brake levers. The rollator is also equipped with a parking brake function. To engage the parking brake push the brake levers down. To disengage the parking brake, pull the brake levers back to a neutral position.

WARNING! Always activate the parking brake when not using the rollator! When placing the rollator on a non-plane surface you must always activate the parking brake!

WARNING! The brakes must always be in a locked position when the rollator is parked! Never use the seat if the brakes are not locked! When seated, never lean backwards. It is recommended to sit only at the front part of the seat. Do not sit down on the bag.

Tip-pedal

The tip-pedal makes it easier for the user to pass small obstacles by tilting the rollator backwards. The tip-pedal is built into the frame.

Reflectors

The rollator is equipped with reflectors on all sides including on the push handles, bag and front wheel forks:

Accessories

There are no accessories provided for this device.

4. Technical data

Panther rollator	Art. 318003	Art. 318004
Height of push handles (adjustable) / Height of the product	70,5 – 92,9 cm	77,3 – 100 cm
Recommended user heights	140 – 187 cm	150 – 195 cm
Seat height	55 cm	62 cm
Total width unfolded / folded	67,3 / 22 cm	66,9 / 22 cm
Width between push handles (min / max)	46,5 cm / 49,2 cm	45,1 cm / 47,7 cm
Length	67 cm	66,5 cm
Weight	6,8 kg	6,8 kg
Max. user weight / bag load	150 kg / 5kg	150 kg / 5kg
Brakes	2 lockable with parking function	
Wheel sizes (front and back)	200 x 34 mm	
Seat width	45 cm	45 cm
Seat depth	20 cm	20 cm
Turning circle	84 cm	84 cm
Tube diameter	Cross for frame Ø22 mm/frame Ø26.25x52mm	
Material	Aluminium frame, wheels made of PP and rubber	

Product label

Example of the product label

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. For disinfection, use only disinfectants certified for use with medical devices available locally.

Wheels can be cleaned with hot water with a mild cleaning agent. After cleaning the wheel bearings should be lubricated with oil periodically or when the wheels start squeaking.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

It is recommended to fold the device for transportation. The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
The rollator is difficult to guide or push	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
The brakes do not work or work only on one side	The brake shoes are worn, or the brakes are not adjusted correctly	Adjust the brakes. Replace the brake shoes if needed (→ service center)
The seat does not fit the frame while unfolding the rollator	The locking device is in a wrong position or defective	Check that the locking device is in a correct position. Replace it if needed (→ service center)

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at

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