## SOFTSLEEP / SOFTLSEEP CONTOUR

Art. 279061, 279062, 279063, 279064

**Gebrauchsanleitung**  
**Brugervejledning**  
**User Manual**  
**Manuale**  
**Bruksanvisning**  
**Instrukcja Obsługi**  
**Manuel de l'utilisateur**

The SoftSleep pillow and SoftSleep Contour is made of high-density, open-cell PU foam also called "Slow Foam" or "Memory Foam". This material has unique properties: it reacts to body weight and temperature fluctuations.

Ergonomic design for maximum comfort

The SoftSleep pillow cover can be easily removed by unzipping it and pulling it off the pillow.

The SoftSleep pillow cover can be washed using a washing machine (the zipper should be closed)

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## 1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

### Intended use of the device

The SoftSleep orthopedic pillow is a medical device, designed for the persons with cervical spine diseases, for positioning of the head and neck during sleep, to reduce discomfort of the cervical spine. The cushion is intended for use at home and in nursing or care institutions. The pillow is only intended for the purposes described above and any other use of the product is prohibited.

### Indications

SoftSleep orthopedic pillows are indicated for use by people with cervical spine disorders which cause neck, back and head aches, muscle stiffness and other disorders.

### Contraindications

Contraindications cannot be linked to a specific clinical picture, but the initiation of the use of the product, as well as the choice of the right version, should be consulted with a specialist and the safety recommendations must be followed. In addition, observe the safety information in Chapter 2.

### Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

### Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

## 2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- Always check that the label is readable and that the product is in good condition without signs of tear and wear such as loose seams, frays and material inconsistency.
- Do not leave the product unattended on the floor, a person may accidentally step on it with the risk of injury.
- Clean the device regularly according to chapter 5.

### Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Orthopedic pillow;
- ✓ 1 User manual;

### Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

## 3. Description

### Product description

SoftSleep pillows are available in 2 shapes (classic, contour) and 2 sizes (M, L) to allow the best personal match. The pillow is made of a high density visco-elastic polyurethane foam with an open cell structure, known as "Slow Foam" or "Memory Foam". The material is thermo-reactive, and adapts to the shape of the neck and the head upon receiving body heat, reducing the pressure of the body on the pillow which helps eliminate typical after-sleep pains. Excellent support for neck and shoulder muscles which relieves them, helps to reduce stress related problems and can help reducing insomnia. All these properties improve the quality of sleep.

The pillowcase is specifically designed as a foam cover, and helps to distribute heat as well as providing a hygienic functions because of its anti-fungal and anti-bacterial properties. There are no contraindications for it to be used by people with asthma, skin diseases or allergies.

### Assembly description

This device doesn't require an assembly instruction.

## Function description

### Preparations for use

Due to the shape and special properties of the pillow material, within the first 2 days to 2 weeks the products might feel uncomfortable depending on the user, it is an adjustment period and normal. After the period of 2 weeks the foam becomes softer and darker as its responsiveness to heat increases. New pillows may emit a certain smell when first unpacked, which disappears within a few days.

### Usage

When you sleep on your side or back, your shoulder should be as close as possible to the pillow so that your head and neck are comfortably supported, as shown on the packaging drawings. When the pillow is heated by body heat, the surface becomes softer. This allows the pillow to flex and shape up evenly to your body. As the pressure is released and the material cools, the pillow material slowly returns to its original shape. If the firmness seems different than usual, please check that the pillow has not been heated in the sun.

### Accessories

There are no accessories provided for this device.

## 4. Technical data

SoftSleep	Art. 279061	Art. 279062	Art. 279063	Art. 279064
Dimensions (L x D x H)	50 x 30 11.5 cm	54 x 32 x 12 cm	50 x 30 x 10 cm	56 x 35 x 12 cm
Material	"Memory Foam"	"Memory Foam"	"Memory Foam"	"Memory Foam"

## Product label

Example of product label sewed on the SoftSleep pillow

### Meaning of the icons

	QR-code link to homepage		CE marked product		Drip-dry
	Manufacturer		Manufacturing date		Medium ironing temperature
	Observe the user manual		Max load		Washing temperature
	Serial no.		Medical Device		Do not tumble dry
	Item Number		Only to be used indoor		Handwash

Example of product label on the box

## 5. Care and maintenance

*Example of the product label on the box*

The outer pillowcase is removable and machine washable in 30°C water. Please refer to the washing instructions on the label of the cover. The zipper of the pillowcase should be closed when washing. Do not use a tumble dryer to dry the pillowcase and air dry naturally.

The foam itself cannot be soaked and washed in water, as this will affect its functioning. Use only damp cloths to clean the pillow and dry naturally after cleaning. Do not wash, iron, or blow dry with hot air.

**Recommended service life:** Normally 5 years, depending on use.

**Storage:** We recommend storage in a dry place in temperatures above 0 °C.

## Transportation

The pillow is easily transported and stored due to its small size, but for further convenience it can be put in its original packaging. The pillow must be rolled, as bends and folds can cause partial damage to the foam. Pillows become a little firmer when not in use as their response to temperature changes is slowed.

## Reuse

We recommend that the pillow can be used by one original user only. This product is not intended for transfer to other users.

## 6. Most common problems and solutions

Problems	Possible causes	Solutions
The pillow is softer than usual	Long period of time in heat from the sun	Wait for the pillow to cool down

## 7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

## 8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

## 9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

[mobilex.dk/medias/2022-04/ce\\_softsleep\\_hovedpude\\_279061\\_64\\_10064.pdf](http://mobilex.dk/medias/2022-04/ce_softsleep_hovedpude_279061_64_10064.pdf)

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