TOILET CHAIR WITH TOILET BUCKET

Art. 302013

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

Toilet chair with
toilet bucket

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mobilex.dk/medias/2025-03/ce_302013_18646.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

The toilet chair with toilet bucket is a medical device intended to be used only as seating support by individuals who require such help during mostly independent using the toilet, bathing or showering (the bathroom must be compatible). The device serves a fundamental function and makes the life of both the physically challenged user and the assistant much easier. The device should only be used indoors. Any other use is prohibited.

Indications

The device is indicated for people with functional limitations, people in wheelchairs and other people with limited mobility of the lumbar spine, knees or hips, and for all those who have difficulties performing everyday personal hygiene in standing position. Any use other than stated above is prohibited.

Contraindications

The device is not indicated for those patients who can't maintain proper body balance in a sitting and standing position. The device must not be used if the user has impaired perception. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- The device must only be used on stable surface.
- Use caution when assembling or adjusting any parts of the device.
- Make sure that all the legs are set to the same height.
- Use device only on flat smooth surface with all 4 legs stably placed on the surface.
- Always check stability of device before use.
- Do not stand on the device.
- Make sure that each mounting screw is securely locked before using the toilet chair.
- Do not press any hard objects against the side tubes, as this may damage the side tubes or the cushion.
- Keep hands or fingers away from moving parts to avoid injury.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Main frame incl. the seat and armrests;
- ✓ 4 Legs;
- ✓ 1 Backrest;
- ✓ 1 Toilet bucket and lid;
- ✓ 2 Cushions, one for the backrest and one for the seat;
- ✓ 1 Soap dish;
- ✓ 1 Toilet paper holder;
- ✓ 2 Screws;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

The toilet chair with toilet bucket is designed for individuals who require support during use of toilet or while showering. The chair is equipped with easily removable EVA cushions on the seat and backrest. It has an aluminium frame with lift up armrests, height-adjustable legs, a bucket, a lid and a backrest. It also comes with a soap dish and a toilet paper holder for added convenience.

The device consists of the following elements:

1. Lift up armrest
2. Backrest
3. Seat
4. Lid
5. Bucket
6. Height-adjustable leg
7. Non-slip rubber shoe

*the seat cushion is removed for showcase

Assembly description

To assemble the toilet chair, first you need to place the main frame upside down and insert the 4 legs into the frame by pressing the locking split on each leg and sliding the legs in the designated holes until the locking split pops out. Turn the chair on its legs and attach the backrest by aligning it to the back tube and tightening the 2 screws provided from the back. Then insert the bucket and place the lid into the seat. Attach the soap dish and the toilet paper holder on the sides of the chair, by sliding them in the designated slots. Finally, attach the EVA seat and backrest cushion by pressing them into the appropriate holes.

Function description

Opening / Closing the armrests

To open the armrests press the button on the inside of the armrest tube and at the same time pull the tube upwards.

To close and secure the armrests hold the armrest tube and press it down until you hear a "click" sound. Make sure that the fixing pin popped into the hole under the armrest button.

WARNING! Do not open or close the armrest with excessive force to avoid damaging the parts.

WARNING! Do not put your hands under the armrest when closing the armrest to avoid injuring your hands.

Adjusting the height of the toilet chair

To adjust the height of the toilet chair you need to adjust length of all legs to the desired setting. To adjust the length of a single leg, press the locking split on the inner side of the leg and slide the leg to the desired setting making sure that the locking split pops out. All legs should be adjusted to the same height.

WARNING! Don't adjust the height of the toilet chair when someone is sitting on it and always make sure all locking splits popped out.

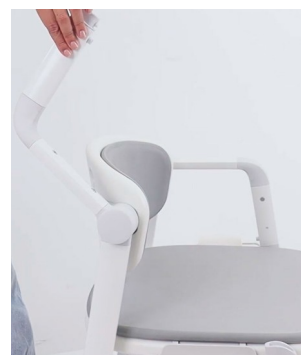
NOTE: The seat height is correctly adjusted when the user is sitting on the toilet chair and both feet fully touch the ground.

Accessories

There are no accessories provided for this device.

4. Technical data

Toilet chair with toilet bucket	Art. 302013
Total size (L x W x H)	51,6 - 58,7 x 64,5 - 67,5 x 72 - 92 cm
Width between the armrests	44 cm
Height	72 - 92 cm
Seat height adjustment range	44,5 - 64,5 cm
Product weight	7 kg
Max. user weight	150 kg

Armrest in the open position

Product label

Example of the product label

	QR-code link to homepage		CE marked product
	Manufacturer		Manufacturing date
	Observe the user manual		Max load
	Serial no.		Medical Device
	Item Number		Only to be used indoor
	Max width		

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. The entire product can be disinfected with a commercially available disinfectant. Always follow the disinfectant manufacturer's instructions for use and safety when disinfecting.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

Problems	Possible causes	Solutions
Toilet chair is not stable	Legs are adjusted to different heights	Adjust all legs to the same height

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

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