**MARLIN WHEELCHAIR**  
Art. 271240, 271244, 271248, 271251

**Gebrauchsanleitung**  
**Brugervejledning**  
**User Manual**  
**Manuale**  
**Bruksanvisning**  
**Instrukcja Obsługi**  
**Manuel de l'utilisateur**

The „Marlin“ wheelchair

Easy folding

Adjustable, detachable  
footrests

Adjustable armrests

**CE**

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## 1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

### Intended use of the device

Intended use of the wheelchair is to move around in a sitting position, independently or with the help of an attendant. Using the wheelchair for purposes other than those indicated above, is prohibited.

### Indications

Manual wheelchairs are indicated for people with mobility limitations who have problems with walking, standing or sitting without stable support.

### Contraindications

The wheelchair should not be used by people who cannot maintain a stable sitting position. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

### Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

### Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: Telefax: Email:	+45 87 93 22 20 +45 87 93 17 77 info@mobilex.dk
-----------------------------------------------------------------	------------------------------	-------------------------------------------------------

## 2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- The device must only be used on stable surface.
- When adjusting the device the user must take care not to squeeze the fingers or other body parts between components.
- Ensure that all 4 wheels are always in contact with the surface during use. This will ensure proper wheelchair balance and will help avoid accidents.
- Using the wheelchair after exposing it to direct and prolonged contact with the sun may lead to skin burns.
- It is not recommended to use the wheelchair on the beach, in mud or in extreme weather conditions.
- The attendant of the wheelchair user should be physically capable and completely healthy.
- Avoid leaving the wheelchair near sources of heat and open flames (campfires, stoves). Smoking is also not recommended during use as the wheelchair is not fire resistant.

### Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Main frame with installed: upholstered back and seat, 2 armrests, 2 drive wheels, 2 brakes, 2 steering wheels, quick release axles, 2 anti-tips (optional).
- ✓ 2 Detachable footrests;
- ✓ 1 User manual;

### Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

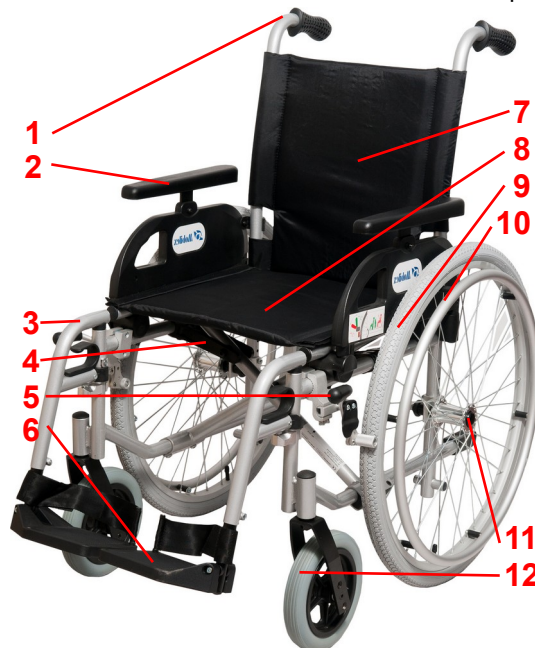
### 3. Description

#### Wheelchair provides:

- A comfortable transportation and storage thanks to its crossed frame structure and wheels with a quick-release axle
- Easiness of maneuvering
- Easy transfer to - and out of the wheelchair
- The durability and the solidity of the structure
- The wheelchair is made of corrosion-resistant powder-coated steel.

#### The device consists of:

1. Push handles
2. Armrest
3. Footrest
4. Cross (under the seat)
5. Parking brake
6. Footplate
7. Backrest
8. Seat
9. Main wheels
10. Push rims
11. Quick-release axle
12. Front wheels

**WARNING!** Always apply the brakes on both sides of the wheelchair.

**WARNING!** The parking brakes are not designed as driving brakes and only serve to lock the wheelchair (park position). They may not be used for braking while driving!


**WARNING!** Do not put fingers between the wheel and the push rims. It threatens the injury of the palm.

#### Assembly

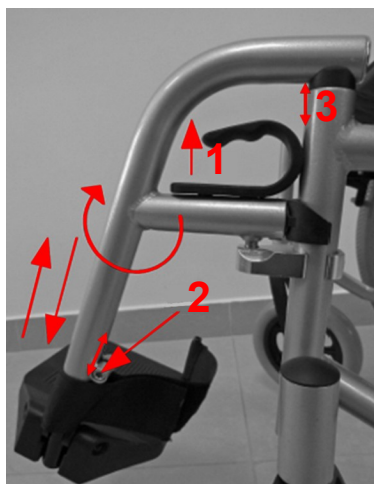The only elements removed from the device for delivery are the footrests. Insert them according to the **Footrests** section in **Function description** below.

#### Function description

##### Footrests

Standard footrests can be disassembled or pushed away on side.


To rotate and disassemble the footrest, first unblock it by pulling up the safety-lock lever (1), then turn the footrest outside and pull it upwards (3). The footrest assembling should be performed in reverse order by inserting it in base of frame (3) and turning it frontwards. The safety-lock will block automatically.

The wheelchair is equipped with height-adjustable footrests. To adjust their length one should loosen a bolt (2), adjust the length of the footrest by sliding the lower tube up or down, and then retighten the bolt.

During using the wheelchair outside, the footrests must be placed on height of minimum 4-5cm from the base. In addition, the footrests possess the ability of flipping the footplates upwards, which facilitates getting in and out of the wheelchair and also disassembling the footrests.

**WARNING! You should never climb the footrests, it can cause falling over the wheelchair. You should never raise the wheelchair using the footrests.**
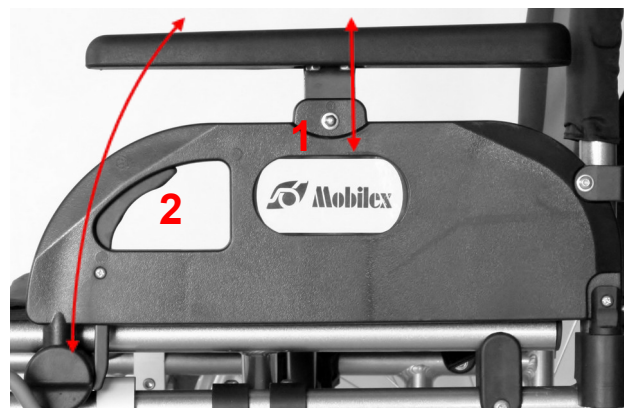
##### Armrests

The armrests can be adjusted in height, as well as be tilted backwards, behind the backrest, to facilitate the transfer of a passenger from or onto the wheelchair.


To adjust the height of the armrest, unscrew and remove the armrest bracket mounting screw (1). Adjust the armrest to the desired height, then insert and tighten the mounting screw back into the armrest bracket.

To tilt the armrest backwards, one should press the lever (2) and lift the armrest up.

To reattach the armrest, simply push it back down until the safety-lock will block itself automatically.

**WARNING! It is prohibited to use the armrests as wheelchair carrying handles.**

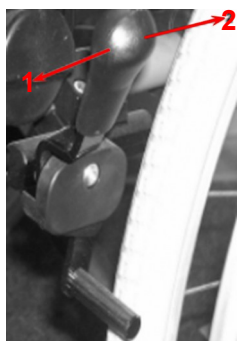**Parking Brakes**

The wheelchair is equipped with parking brakes on both rear wheels to prevent rolling away.

To apply the brake push the lever forward (1). To release the brake pull the lever towards the wheel (2).

Always apply the brakes on both sides of the wheelchair.

**WARNING!** The parking brakes are not designed as driving brakes and only serve to lock the wheelchair (park position). They may not be used for braking while driving!

To set the correct distance between brake and tire:

Using a cross head screwdriver remove the 2 screws (1) that secure the brake to the underside of the guide bar.

Slide the brake backwards or forwards to the correct position (approximately 10mm between the tire and the released brake shaft).

Then tighten the 2 screws.

**24" rear wheels**

The wheelchair is equipped with 24" main wheels with puncture-resistant polyurethane tyres. The wheelchair is delivered with the standard push rims. The material and the distance between the push rim and the wheel can influence on the grip strength of the wheelchair's user.

One should be careful and not allow the fingers to get between spokes during driving in narrow passages. To avoid such risk it is recommended to use a spoke protection.

The wheels on "Marlin" chair can additionally be easily removed and fitted in by pressing the quick-release button placed in the center of the wheel (1).

To install the wheel:

Place the end of the wheel axle back into the socket (2), press and hold the button (1), push the axle as far as it will go into the socket and release the button. Check if the wheel is fastened by pulling it outwards several times.

The wheel balance on the "Marlin" can be additionally adjusted by moving the wheel axle. In this case it is necessary to adjust also the position of the turning wheels.

**WARNING!** We recommend such adjustment to be done by the qualified staff. An inappropriate adjustment can cause deterioration of the safety usage of the wheelchair and decrease its durability.

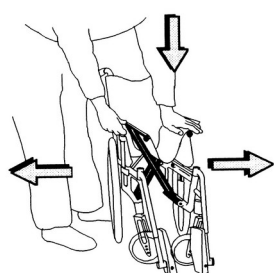
**Safety belts**

There is possibility to attach the hip safety belts. They are not standard equipment.


**WARNING!** To ensure a secure push both handles must be set to the same height.

**WARNING!** The push handles have to be used only for pushing and steering the wheelchair. You should NOT use it for lifting or tilting the wheelchair!

**How to fold and unfold the wheelchair**

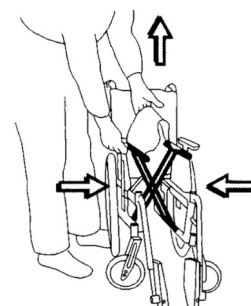Placing one hand on the seat upholstery press one of the tubes, which support the seat upholstery (in accordance with the instructions on the drawing). One should be careful and not put the hand between tubes and the wheelchair frame during unfolding it.

### Tip pedals

The tip pedal on wheelchairs makes it easier for the attendant to avoid obstacles by tilting the wheelchair backwards (see photo left). The tip pedal is built into the frame (see photo right).

**NOTE! DO NOT** tilt the wheelchair with the handles, but use the tip-pedal!

## Advanced settings and adjustments

**WARNING!** We recommend that all advanced adjustments are only carried out by qualified personnel. An unsuitable adjustment may affect safety when using the wheelchair and reduce the service life of the wheelchair.

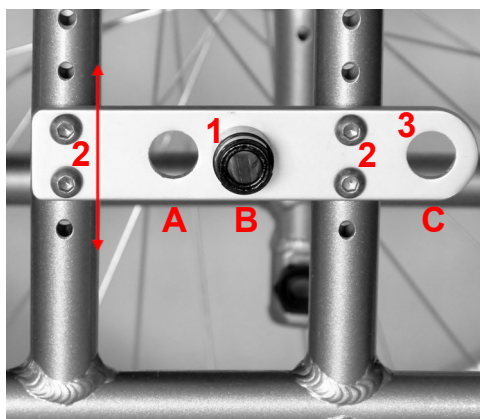
### Frame adjustments - changing the position of the rear and front wheels

The construction of the wheelchair makes it possible to change the height of the seat and also to shift the center of gravity of the wheelchair.

- Adjusting the seat height is done by changing the mounting position of the rear wheels (up/down) and changing the mounting position or size of the front wheels.
- Adjusting the balance of the wheelchair is done by changing the mounting position of the rear wheels (front/rear).

### Adjusting wheel position (vertically and horizontally)

If necessary, the seat height of the wheelchair can be adjusted by raising or lowering the axles of the rear wheels. In this case, the position of the front wheels must also be adjusted.

It is possible to adjust the rear wheels up or down, forwards or backwards to suit the tyre size and the personal preferences of the user. The height adjustment of the wheel axles allows the adjustment of the seat height. The horizontal alignment (front / back) allows to influence the stability and manoeuvrability of the chair. The most common and neutral position is "B". If it is necessary to increase the manoeuvrability of the wheelchair, position "A" can be used. With the wheels in position "A" it is easier to manoeuvre the wheelchair but the stability to the rear is significantly reduced.

If the wheels are set to position "A" the chair should always be equipped with an activated anti-tip. Persons with amputated leg(s) should never use the wheelchair with the wheels in position "A" - the risk of tipping is too high. Leg amputees should set position "C" as it provides greater stability than the other positions. Even when using this setting, the wheelchair should be equipped with anti-tippers (accessories). As the manoeuvrability of the wheelchair is made more difficult with the axles in

position "C", this setting is only recommended for leg amputees.

**WARNING!** The correct position of the wheels should only be determined by professional therapists or specialist companies.

There are 4 different height settings and 3 different horizontal settings possible.


To adjust the axles horizontally (front or back) proceed as follows:

1. Remove the wheels with the quick-release axle.
2. Remove the wheel mounting bracket (1) and place it in the desired position (A, B, C)
3. Repeat the process with the opposite axle.

To adjust the axle height, proceed as follows:

1. Remove the wheels with the quick-release axle.
2. Unscrew the wheel mounting plate by loosening the 4 screws(2).
3. Move the plate up or down to your desired height dictated by screw slots.
4. Fix the axle mounting plate again with the 4 screws.
5. Repeat the procedure with the opposite mounting plate.

### 8" front wheels

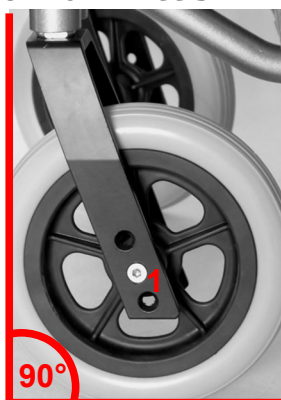
The wheelchair is equipped with PU wheels 200x50 mm as standard. It is possible to mount wheels with different sizes. This might be necessary after adjusting the rear axles. If the vertical position of the rear wheels is changed, it is always necessary to adjust the position of the front wheels accordingly. The vertical axis of rotation of the fork should always be set at a 90° angle to the ground (+/- 2°).

To make the adjustment, move the front wheel axle to another hole by loosening the bolt (1), moving the wheel up or down to the desired height and tightening it again in the selected hole. There are three holes available. In some cases it may be necessary to replace the standard front wheels (8") with a wheel of a different size (6" or 7").

### Anti-tip support (optional)

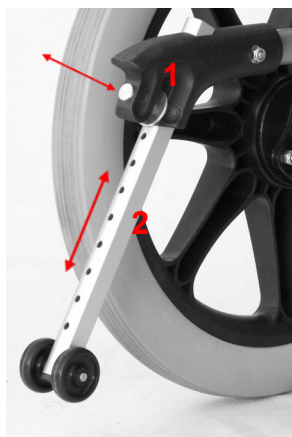This component increases rear stability and safety. It is especially recommended for wheelchairs of which center of gravity has been shifted rearwards. It should be installed on all wheelchairs used by leg amputees and is recommended for wheelchairs with the rear wheels mounted in the forward position.

To activate the support, pull and hold the locking pin backwards (1), then move the support (2) to a desired position. Release the locking pin (1) so that it locks the support in this position.

To deactivate the support, simply move the support all the way back after pulling the pin and let go once adjusted.

The safe distance between the support and the ground should not exceed 4 cm.

**WARNING!** Proper installation of the anti-tip support should only be performed by qualified personnel.

### Accessories

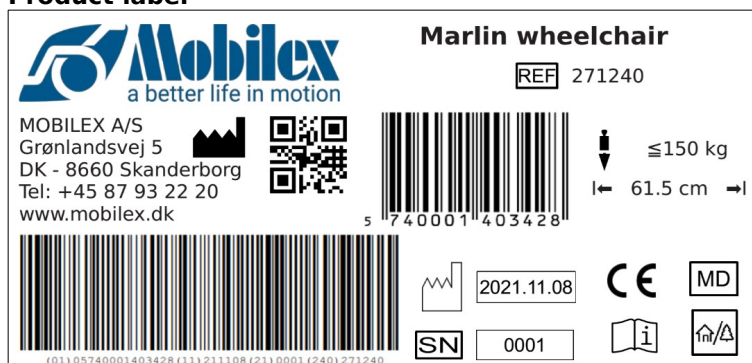
As an option, the wheelchairs can be equipped with the following accessories (not part of the standard equipment)

- ✓ Anti-tip
- ✓ Safety belt

### 4. Technical data

Marlin wheelchair	271240	271244	271248	271251
Seat-width	40 cm	44 cm	48 cm	51 cm
Seat depth	41 cm			
Seat height	50 cm			
Total width (depending on choice)	61,5 cm	65,5 cm	69,5 cm	72,5 cm
Weight (depending on size and accessories)	up to 19,3-21 kg			
Max. user weight	150 kg			

### Product label

Example of the product label

	QR-code link to homepage		Manufacturer
	CE marked product		Item Number
	Observe the user manual		Max load
	Medical Device		Manufacturing date
	To be used indoor and outdoor		Serial no.
	Max width		

Meaning of the icons

## 5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. For disinfection, use only disinfectants certified for use with medical devices available locally.

Wheels can be cleaned with hot water with a mild cleaning agent. After cleaning the wheel bearings should be lubricated with oil periodically or when the wheels start squeaking.

The seat and the backrest are made of a very durable strengthened nylon. The seat upholstery can be easily removed by slackening the screws placed on the frame tubes. The backrest upholstery can be removed by slackening the screws placed on the backrest tubes.

The seat and the backrest upholstery can be washed with use of sponge and gentle washing soap.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

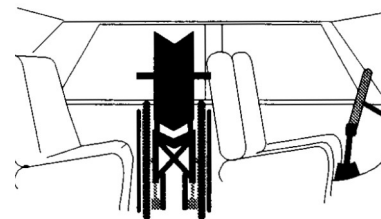
- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

**Recommended service life:** Normally 5 years, depending on use.

**Storage:** We recommend storage in a dry place in temperatures above 0 °C.

### Transportation

It is recommended to transport the device folded, in a car trunk. If that method of transportation is unavailable, the folded wheelchair can be transported behind the front seats of the car (right image).

### Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

## 6. Most common problems and solutions

Problems	Solutions
The wheelchair seems to tilt to one side	<ul style="list-style-type: none"> <li>• check fastening of the front turning wheels</li> <li>• one of the rear tires might be inflated more than the other one</li> </ul>
The wheelchair is hard to push	<ul style="list-style-type: none"> <li>• check fastening of the front turning wheels</li> <li>• remove dirt or entangled hair from the front turning wheels</li> <li>• too large load applied on the front turning wheels: move the center of gravity</li> </ul>
The wheelchair is hard to turn	<ul style="list-style-type: none"> <li>• ensure, that bolts fastening the front turning wheels are not tightened too strong</li> <li>• remove dirt or entangled hair from the front turning wheels</li> </ul>
Brakes are not working correctly	<ul style="list-style-type: none"> <li>• adjust the distance between the brake and tires</li> </ul>
It is hard to fold and unfold the wheelchair	<ul style="list-style-type: none"> <li>• the upholstery fitted too tight</li> <li>• clean up or apply grease to the cross under the seat</li> </ul>
The wheelchair is not stable	<ul style="list-style-type: none"> <li>• ensure, that all screws/bolts and nuts are tightened</li> <li>• ensure, the wheelchair is properly adapted to the passenger</li> </ul>

## 7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

## 8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

## 9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at

[mobilex.dk/medias/2021-11/ce\\_marlin\\_koerestol\\_271240\\_271244\\_271248\\_271251\\_8945.pdf](http://mobilex.dk/medias/2021-11/ce_marlin_koerestol_271240_271244_271248_271251_8945.pdf) or via the QR code on the right.

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