COMMODE AND SHOWER CHAIR

Art. 305001, 305002

**Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur
Manual del usuario**

White powder coated frame

Standard model
w/4 lockable
5" wheels

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mobilex.dk/medias/2025-07/ce_305001_02_19008.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

Use is indicated for people with limited standing ability and reduced or no functional ability of the lower extremities. The commode wheelchair is a medical device intended to be used only as seating support by individuals who require such help during mostly independent bathing, showering and defecating (the bathroom must be compatible). It is also a transport option intended to transport patients from and to bathrooms or toilets. The device serves a fundamental function and makes the life of both the physically challenged user and the assistant much easier. The device should only be used indoors. Any other use is prohibited.

Indications

The commode and shower chair with four 5" wheels is indicated for people with mobility limitations who have difficulty walking, standing, or sitting without stable support, possibly as a result of structural and/or functional damage to the lower limbs (e.g. amputation, consequences of injuries, musculoskeletal/neuromusculoskeletal movement disorders). These assistive products are dedicated to people who, due to their disability, cannot perform their daily hygiene activities in regular shower cubicles or bathtubs and who are unable to use regular toilets, even after modifications.

In addition, because of the four 5" wheels: Inability to propel a wheelchair (e.g. in case of limited strength and grip function or lack of arm/hand coordination).

Contraindications

The Commode and shower chair should not be used by persons who cannot maintain a stable sitting position. Users with cognitive impairment must be monitored during use of the device and it is recommended that an individual assessment of the user's health condition is carried out before use. The selection of the orthopaedic device that is suitable for a given medical condition must always be made with the help of a specialist doctor or physiotherapist. In addition, you must comply with the safety information in chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Moblex A/S	Phone:	+45 87 93 22 20
Grønlandsvej 5	Telefax:	+45 87 93 17 77
DK-8660 Skanderborg	Email:	info@mobilex.dk
Denmark		

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- Use caution when assembling or adjusting any parts of the device.
- The device must not be loaded with more weight than indicated in Technical data.
- All wheels MUST be in contact with the floor at ALL TIMES during use. This will ensure that the device is properly balanced and should decrease the risk of accidents.
- The device must only be used on a stable surface.
- When using the device in a stationary position, the brakes MUST be engaged.
- Do not stand on footrests, and do not sit on the edge of the seat with the legs on the footrests
- The device is only to be used indoors and on flat surfaces without any obstacles.
- Always maintain a stable sitting position when using this device and users should not be left on the device unattended. Remove the seat cover only when chair is used for going to the toilet.
- If the seat cover is placed over the seat hole, it is absolutely necessary to also place the bucket WITH THE LID under the seat.
- The commode and shower chair must not be used to transport people or goods.
- Never stand on the footplates when getting on or off.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Main frame including 4 wheels equipped with parking brakes;
- ✓ 1 Detachable seat and seat cover;
- ✓ 1 Detachable backrest frame with armrests;
- ✓ 2 Detachable footrests;
- ✓ 1 Sanitary bucket with a lid;
- ✓ 1 Set of tools for mounting;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

The Commode and shower chair comes with: replaceable sanitary bucket, soft PU seat, parking brakes, detachable footrests, flip-up armrests etc. The product is made of materials that are corrosion and disinfectant resistant. The frame is made of powder-coated aluminium tube.

The device consists of the following elements:

1. Soft PU backrest
2. Flip up armrest
3. Back frame assembly knob
4. Detachable sanitary bucket
5. Steering castor with parking brake
6. Detachable seat cover
7. Soft PU seat
8. Legrest
9. Footrest detaching lever
10. Legrest length adjustment bolt
11. Lifiable footplate

Assembly description

- Put the main frame upside down on the floor. You'll see four holes at the ends of vertical frame tubes.
- Insert the wheel assemblies into the tubes and fix them in desired setting using provided bolts. Check if they are attached correctly by trying to pull them out. If they slide around, check that the bolt went through both the frame and wheel assembly and tighten the bolt again.
- Put the frame on its wheels and lock the wheel brakes by pushing them down with the tip of a foot to make further assembly safe.
- Unscrew both frame assembly knobs.
- Insert the backrest tubes into holes on the rear end of main frame. Tighten the knobs on both sides of the chair and check that the backrest is stable.
- Put the lid on the sanitary bucket and slide it into the guide rails under the seat from the rear of the chair. Once the bucket is in place, place the PU seat on the frame by pressing the seat clamps on the frame and add the detachable seat cover on the PU seat. Seat and seat cover can be replaced.
- Put the top end of the footrest into the hole in the front of the main frame, and turn the footrest towards the front of the frame until it's locked. Unfold the footplate and repeat for the other footrest.

Pic. 1: Mounting the backrest

Pic. 2: Mounting the seat

Pic. 3: Mounting the footrest

Function description

WARNING! Always engage the brakes when a user is sitting down on or getting up from the chair, leaving the chair with a user in it, or putting the chair away, especially on slopes.

Using the armrests

The armrests can be easily lifted up and pushed back down. This feature can help make it easier for the user to get in and out of the wheelchair. Raising the armrests makes it easier to perform personal hygiene while showering.

WARNING! Never leave the chair unattended with armrests lifted when there is someone in it.

Footrest functionality

- To adjust the length of the footrest, loosen the footrest adjustment bolt. Move the footrest to the desired length and tighten the bolt. Repeat for the other footrest.
- To remove the footrest, pull the footrest detaching lever. The footrest can be removed when turned away by pulling it upwards. Repeat for the other footrest.
- The footplates can be folded upwards.
- These features can make it easier for the user to get in and out of the wheelchair, while also allowing for easier personal hygiene while showering.

WARNING! Never move the chair with footrests detached when the user is on it. Always put user's feet on foot plates when moving the chair.

WARNING! The length of footrests must be adapted to the height of the user.

NOTE: The length of footrests is correctly adjusted when while sitting the user sits on the wheelchair, the thighs rest flat on the seat and both feet rest on the footplates.

Using the sanitary bucket

The Commode and shower chair is designed for people who have difficulty using regular toilets.

To use the bucket in the Commode and shower chair, remove the seat cover and put the bucket (without the lid) in the chair. After use remove the bucket from the wheelchair again and put the lid back on the bucket. Clean the bucket before another use. Put the bucket back into the chair and put the seat cover back in the seat. If you want to replace the sanitary bucket, it can be purchased as a spare part (item number: 305026)

Using the device with a standard toilet bowl

To use the toilet with the device, remove the toilet bucket, then move the chair backwards over the toilet bowl and remove the seat cover.

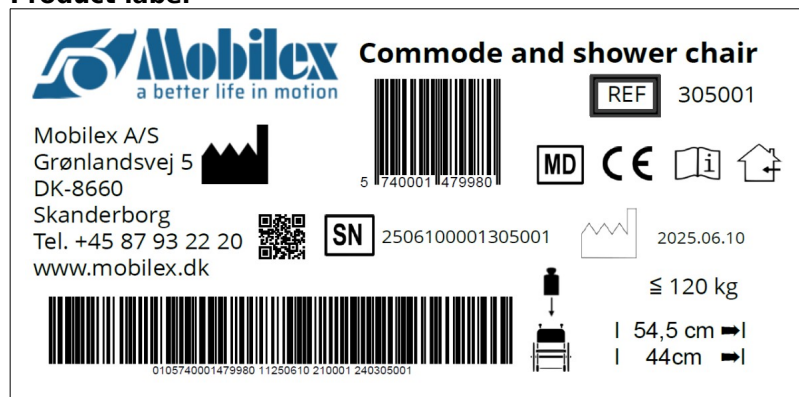
Accessories

There are no accessories provided for this device.

4. Technical data

Commode and shower chair	Art. 305001	Art. 305002
Seat width	44 cm	44 cm
Seat depth	40 cm	40 cm
Seat height (Front/back)	53,5 / 52,5 cm	53,5 / 52,5 cm
Seat opening	19 x 26 cm	19 x 26 cm
Pass over height	44 cm	44 cm
Pass over width	34,5 cm	34,5 cm
Seat/Back angle	90 °/ 12°	91 °/ 12°
Distance between armrests	43,5 / 44,5 cm	43,5 / 44,5 cm
Backrest height	41 cm	41 cm
Total width	54 cm	54 cm
Total depth	89 cm	89 cm
Total height	92 cm	92 cm
Max. user weight	120 Kg	120 Kg
Weight	12,60 Kg	12,60 Kg

Product label

Example of the product label

	QR-code link to homepage		Manufacturing date
	Manufacturer		Loading force
	Observe the user manual		Max width
	Only to be used indoor		serial no
	CE marked product		Item Number
	Medical Device		

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. The entire product can be disinfected with a commercially available disinfectant. Always follow the disinfectant manufacturer's instructions for use and safety when disinfecting.

Wheels can be cleaned with hot water with a mild cleaning agent. After cleaning the wheel bearings should be lubricated with oil periodically or when the wheels start squeaking.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

Cleaning the bucket

The bucket must be emptied and disinfected after use. Do not use water with at temperature higher than 80°C.

When using a steam cleaner/autoclave: Remove the handle from the bucket and clean the bucket after it has been laid flat down. Do not use the steam cleaner/autoclave on the bucket for more than 10 minutes. Do not place the bucket at an angle to avoid deformation during cleaning.

WARNING! If the product is exposed to very calcareous water, a binding and accumulation of lime on wheel bearings and other moving parts may occur. In this case, the product can be cleaned with a limescale remover and then lubricated.

WARNING! When used in highly chlorinated environments extra strong corrosion may occur and the warranty will be void.

WARNING! If the frame is bent or broken, the product must not be regenerated but must be replaced.

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

It is recommended to disassemble the device for transportation. The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

If the device needs to be carried without being fully disassembled, it is recommended to remove the footrests and lift the device by grabbing it by the seat tubes on the sides.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

Problems	Possible causes	Solutions
The wheels do not turn smoothly and easily	The space between wheels and forks is filled up with dirt	Clean the space between wheels and forks of all unintended objects and elements

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

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Mobilex A/S ofrece una garantía de 2 años por defectos de fabricación o fallos en los materiales.