


SCHINE WALK

A support you can rely on

Have you ever thought about the small rubber tip at the end of crutches and canes? What its actual function is, and if it can be replaced? Maybe you've wondered what a new design would look like - and if it would make a difference?

We have the answer, Schine Walk. A lightweight crutch tip that makes walking with crutches or a cane both safer and smoother, while providing a good grip at various angles and on different surfaces.

89g

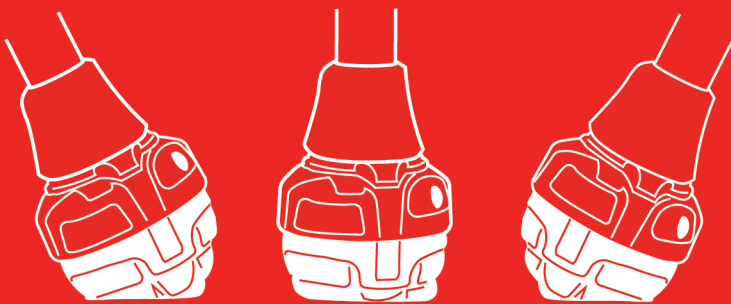

Reliable, stable and flexible



The bowl-shaped lower part of Schine Walk, the shock absorber, is made of a high-friction plastic.

When weight is applied, the shock absorber compresses, creating a large contact surface that adapts to the ground.

The design of the shock absorber is inspired by tire treads, contributing to a good grip in various environments.

Contact Surface

Compared to a standard crutch tip, Schine Walk has a larger contact surface. It is wide and flexible, allowing it to adapt to various angles and surfaces, such as uneven terrain or sloping ground.

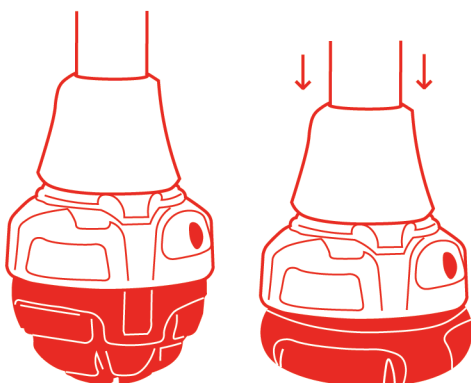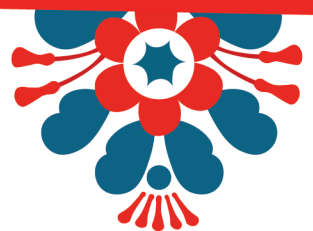

Schine Walk




A standard crutch tip

The concept Schine Walk

The holder, to the right, is attached to the crutch using a nut. The shock absorber, below, is available in different versions: medium and firm. For winter surfaces, the firm shock absorber is equipped with studs from Bestgrip®. The shock absorbers are color-coded.

The shock absorber is easy to place and remove from the holder, making it quick and convenient to switch shock absorbers without having to remove the holder. This is practical when switching to surfaces with snow and ice. between indoor and outdoor surfaces—just swap to the studded version when heading outside.

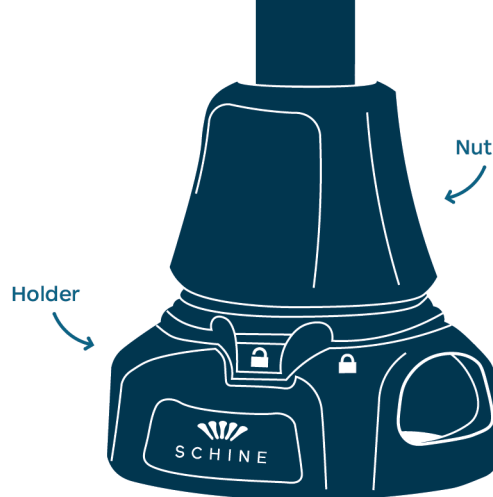

Shock absorber - Firm, red

- More stable with firmer shock absorption.
- Ideal for those who need extra resistance.
- For example, when relying on one leg for support or when having a heavier stride, such as during training.



Shock absorber, Medium, blue

- Soft and flexible resistance.
- Suitable for those who do not put heavy pressure on a crutch or cane while walking.
- For example, as support for balance or when walking with a prosthesis.



Shock absorber with studs, black

- The firm shock absorber is equipped with 9 studs from Best Grip®.
- With its grip and large contact surface it provides extra security when walking on ice and snow.

Quick product facts



Schine Walk...

Mounted it extends by 6 cm.

Fits shafts with a diameter of 17mm, 19 mm & 19,5mm.

Is 95mm long.

Is 56mm wide.

Made in Sweden of recyclable plastics.

Has a maximum user weight of 140kg.

Is patented.



Scan for more information.



Find a retailer at: mobilex.dk

Milmas AB
Lägervägen 8
SE-791 52 Falun

