Jaguar LIGHT Rollator

Art. 317410, 317411, 317412, 317414, 317415, 317416, 317417

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur
Gebruikershandleiding
Návod k použití
Návod na použitie

Indoor rollator
7" black wheels

Jaguar standard

SUPREME
7" silver wheels

Jaguar standard, folded

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mobilex.dk/medias/2026-02/jaguar_light_ce_20006.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

This product is medical device which has been designed to increase the mobility of persons who have difficulties standing or walking. The rollator enables those persons to regain the ability to perform daily activities. The rollator is suitable for persons who as a result of various diseases need stable walking support. The rollator should only be used as a support for standing and walking. Any other use is prohibited.

Indications

The rollator is a medical device designed to help people with functional limitations. The indications for use are diseases that cause mobility limitations and disabilities, obstructing independent sitting, standing and walking. Therefore, there is a need for an additional stable support when performing the activities listed above. Indications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status by professional physician or physical therapist, but main target user groups are elderly people, persons with paresis, rheumatic diseases, and other people with limited mobility of the lumbar spine, knees or hips. Using the product for any purpose other than those described above is prohibited.

Contraindications

Rollators are not indicated for those persons who can't maintain proper body balance in a standing position. The device must not be used if the user has problems with perception. Contraindications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S	Phone:	+45 87 93 22 20
Grønlandsvej 5	Telefax:	+45 87 93 17 77
DK-8660 Skanderborg	Email:	info@mobilex.dk
Denmark		

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device and the bag must not be loaded with more weight than indicated in Technical data.
- All wheels MUST be in contact with the floor at ALL TIMES during use. This will ensure that the rollator is properly balanced and should prevent accidents.
- Use caution when assembling or adjusting any parts of the device.
- Always check if the brakes function properly before use.
- Use the rollator only when it has been fully unfolded and the locking device has been activated.
- The rollator should only be used indoors on a flat stable surface.
- Parking brakes must always be engaged when the rollator is not in use.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Rollator;
- ✓ 1 Tray;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

The "Jaguar light" rollator has a powder coated aluminium frame. It offers great manoeuvrability, stability and has a height adjustable push handle to fit the user. The brakes can be used while walking or as parking brakes. The basket eliminates the need to hand carry items.

The rollator consists of the following elements:

1. Push handle
2. Brake handle
3. Handle height adjustment clip
4. Folding lock button
5. Brake
6. Rear 7" wheel
7. Parking brake clip
8. Tray
9. Pocket with a zipper
10. Basket
11. Front 7" turning wheels

Assembly description

This device doesn't require an assembly instruction.

Function description

Folding/unfolding the frame

To unfold the frame pull the rear frame part backwards. Gently press down the handle until the frame fully unfolds and "clicks", which indicates the rollator is locked in an unfolded position. The tray can now be placed on the basket. To fold the frame, take the tray off and press the folding lock button. Pull the rear frame part inwards.

Push handle height adjustment

To adjust the height of the push handle, unlock the height adjustment clips on both sides and slide the handle to the desired setting. Lock the height adjustment clips. Both sides of the handle should always be at set to the same height.

Brakes

Brakes can be gradually activated by pulling the brake handle with increasing force. Brakes are fully engaged when brake handle can be locked with the parking brake clip and the rollator is fully stopped. To stop braking let go of brake handle.

The rollator is also equipped with a parking brake function. To engage the parking brake pull the brake handle and lock it in a pulled position with the parking brake clip. To disengage the parking brake, unlock the brake handle and move it back to a neutral position.

Push down parking brake handle (model 317415)

On Jaguar SUPREME the parking brake is engaged by pushing down the brake handle and unlocked by pulling it back to a neutral position.

WARNING! Always activate the parking brake when not using the rollator! When placing the rollator on a non-plane surface you must always activate the parking brake!

Accessories

There are no accessories provided for this device.

4. Technical data

Jaguar LIGHT rollator	Art. 317410-12+14+16-17 Various colours	Art. 317415 SUPREME
Height (adjustable push handle)	83,5 - 94,5 cm	90,3 - 96 cm
Folded rollator size (H x W x D)	83,5 x 56 x 26 cm	90,3 x 56 x 26 cm
Width	56 cm	56 cm
Length (depth) unfolded / folded	66 / 26 cm	66 cm / 26 cm
Weight	5,84 kg	
Max. user weight / bag load	100 kg / 5 kg	100 kg / 5 kg
Brakes	2 lockable with parking function	
Wheel sizes	Front and back: 7" (175 x 27 mm)	
Turning circle	80,8 cm	
Tube diameter	Handle tube Ø21*1,5*402mm, Front connecting tube: Ø19*1*398mm, Fixed tube rear frame 19*1*395mm, Frame front Ø 18,5x 36,3/frame back Ø26.5x39,3mm	
Material	Aluminium frame, plastic (PA+GF), basket made of fabric	

Product label

Example of the product label

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	Only to be used indoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. For disinfection, use only disinfectants certified for use with medical devices available locally.

Wheels can be cleaned with hot water with a mild cleaning agent. After cleaning the wheel bearings should be lubricated with oil periodically or when the wheels start squeaking.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

It is recommended to fold the device for transportation. The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

Problems	Possible causes	Solutions
The rollator is difficult to guide or push	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
The brakes do not work or work only on one side	The brake shoes are worn, or the brakes are not adjusted correctly	Adjust the brakes. Replace the brake shoes if needed (→ service center)

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

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