AVANTGARDE + SERVAL ROLLATOR

Item nr. 312801 | 312802 | 312803 | 312804

Gebrauchsanleitung | User Manual | Brugervejledning
Bruksanvisning | Manuel de l'utilisateur | Manuale d'uso
Manual de usuario | Bruksanvisning | Gebruikershandleiding | Instrukcja użytkowania

Mobilex A/S
Grønlandsvej 5
8660 Skanderborg, Denmark
www.mobilex.dk

Mobilex Sp. z o.o.
ul. Duńska 3/5
91-204 Łódź, Polen
www.mobilex.pl

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1. Introduction

You have chosen a high-quality Mobilex product and we are particularly pleased about that. This user manual contains a description and important information about the use of the medical device. It is intended to help you use the product correctly and safely. It is very important that you read the manual carefully before using the product. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly striving to improve and update its products. Therefore, we reserve the right to change specifications and technical data of the product without further notice.

Intended Use

This product is a medical device designed to increase the mobility of persons who have difficulty standing or walking. The rollator enables these persons to regain the ability to perform daily activities. The rollator is suitable for persons who require a stable walking aid due to various illnesses. The rollator should only be used as a support when standing and walking. The rollator seat can be used for sitting and resting. Any other use is prohibited.

Indications

The rollator is a medical aid that helps people with functional limitations. The indications for use are diseases that lead to mobility limitations and disabilities that impede independent standing and walking. Therefore, additional stable support is required when performing these activities. The indications cannot be related to a specific clinical picture and depend on the individual assessment of the health status of the potential user by a physician or physiotherapist. However, the main target groups are elderly people, persons with paresis, rheumatic diseases and other persons with limited mobility of the lumbar spine, knees or hips. The use of the product for any purpose other than that described above is prohibited.

Contraindications

Rollators are not suitable for persons who cannot maintain their balance while standing. The product must not be used if the user has problems with perception. Contraindications cannot be related to a specific clinical picture and depend on the individual assessment of the health status of the potential user. The choice of the appropriate orthopaedic aid for a specific clinical picture must always be made with the help of a specialist physician or physiotherapist. In addition, please observe the safety instructions in Chapter 2.

Areas of Use

The product is suitable for indoor and outdoor use and can be used by private individuals in the home environment, in home care, in health and care facilities and in hospitals.

Medical Incidents and Adverse Events

Should adverse events occur in connection with the product, please contact the dealer or the manufacturer:

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark

Tel: +45 87 93 22 20 | Fax: +45 87 93 17 77 | Email: info@mobilex.dk

2. Safety Requirements

Ensure that this user manual has been read by all persons who use the product. The manufacturer assumes no responsibility for damage and/or injuries caused by failure to observe the user manual.

- Use the product only in perfect condition.
- If defects or faults are detected, you must immediately take the product out of service and contact the dealer.
- Follow all recommendations and warnings from the manufacturer.
- Observe the information on the product label.
- Use the product only for the described purpose.
- Do not climb on the product.
- Do not make any structural changes to the product unless you have written permission from the manufacturer for such modifications.
- The product and bag must not be loaded with more weight than specified in the technical data.
- All wheels **MUST ALWAYS** be on the ground during use. This ensures that the rollator is properly balanced and reduces the risk of accidents.

- Be careful when assembling or adjusting parts of the device.
- Always check that the wheel brakes are functioning properly before use.
- Only use the rollator when it is fully unfolded and the locking device is activated.
- The seat may only be used when the parking brakes are activated.
- Both brakes should always be activated or locked at the same time.
- It is not permitted to carry a person or an object on the seat while the rollator is in motion.
- The parking brakes must always be engaged when the rollator is not in use.

Delivery

The product is delivered in a cardboard box. The contents of the package are:

- 1 rollator folded incl. mounted seat, back strap and shopping bag
- 1 user manual

Preliminary Inspection

Check the contents of the box against the above list. If at this point you find that a part is missing or damaged, contact your supplier immediately.

3. Description

Product Description

The Avantgarde/Serval Rollator has 4 wheels and a powder-coated carbon frame. Thanks to the lightweight frame, it can be easily folded and offers simple storage and easy transport. The rollator runs smoothly and is sufficiently stable. The push handles can be adjusted in height. The rollator is supplied with a mounted seat, a back strap and a shopping bag.

The rollator consists of the following elements:

1. Push handle (Material: PP+TPR)
2. Brake lever (Material: PP)
3. Seat tube (Material: Alu.)
4. Button for handle height adjustment (Material: PP)
5. Brake (Material: PP)
6. Rear wheel (Material: PP+TPR)
7. Tilt pedal (Material: carbon)
8. Back strap (Material: Fabric)
9. Folding lock clamp (Material: PP)
10. Seat with strap for folding (Material: Fabric)
11. Shopping bag (Material: Fabric)
12. Swivel front wheel (Material: PP+TPR)
13. Reflectors

Assembly Description

No assembly instructions are required for this product.

Description of Function

Folding/Unfolding the Frame (Img. 2 - 5)

To unfold the frame unlock the folding lock clip and press down on the seat tubes until the frame unfolds and "clicks", which indicates the rollator is locked in an unfolded position. To fold the frame pull up the folding strap and finish folding the frame by pushing seat tubes towards each other and locking them together with the folding lock clip.

⚠ The seat tubes must lock firmly into place on both sides when unfolding the rollator. All parts must be securely locked before the rollator can be used. Failure to observe these warnings may result in injury.

Push Handle Height Adjustment

To adjust the height of the armrests, press the height adjustment pin and move the armrest up or down by one step. Repeat the process until the desired setting is reached, and repeat the process for the second handle.

NOTE: Both handles should always be at the same height. The correct height of the push handles should always be at wrist height of the user when standing with arms at the side of the body.

Brakes

The brakes can be activated gradually by pulling the brake levers upwards with increasing force. The brakes are fully activated when the brake levers touch the handles and the rollator has come to a complete stop. To end the braking process, release the brake levers. The rollator is also equipped with a parking brake function. To activate the parking brakes, push the brake lever downwards until it locks in this position. To release the parking brakes, pull the brake levers back to the neutral position.

⚠ Always activate the parking brake when you are not using the rollator! If you park the rollator on an uneven surface, you must always activate the parking brake!

⚠ The brakes must always be locked when the rollator is parked! Never use the seat when the brakes are not locked! Never lean back when sitting. It is recommended to sit only on the front part of the seat. Do not sit on the bag.

Tilt Pedal

The tilt pedal makes it easier for the user to overcome small obstacles by tilting the rollator backwards. The tilt pedal is integrated into the frame.

Reflectors

The rollator is equipped with reflectors on the push handles and on front of the tubes for the bag.

Accessories

There are no accessories for this device.

4. Technical Data

Parameter	Art. 312801 / 312803	Art. 312802 / 312804
Push handles height adjustable	70 – 87 cm	76 – 99 cm
Recommended for user height	140 – 187 cm	150 – 195 cm
Seat height	55 cm	62 cm
Folded rollator size (H x W x D)	81,5 x 22 x 65 cm	84 x 22 x 65 cm
Total width open	63 cm	63 cm
Length open / folded	63 / 22 cm	63 / 22 cm
Weight (rollator / bag)	5,5 / 0,6 kg	5,5 / 0,6 kg
Max. load capacity (rollator / bag)	150 kg / 5 kg	150 kg / 5 kg
Brakes	2 lockable with parking function	2 lockable with parking function
Wheel sizes (front and rear)	200 x 36 mm – PA+TBE	200 x 36 mm – PA+TBE
Turning circle	80,8 cm	80,8 cm
Material	Carbon frame, aluminium cross, plastic	Carbon frame, aluminium cross, plastic

Product Label (Img. 6)

The product label is located on one of the lower tubes of the main frame. It contains important information: name, serial number, production date, article number, maximum user weight, maximum bag load, product width and manufacturer data. The label must never be removed from the device.

5. Maintenance & Care

Normal dirt can be removed with commercially available cleaning agents and a sponge or soft cloth. Check the specific product information and use only commercially available cleaning products that are suitable for

cleaning. For disinfection, use only disinfectants that are certified for use with medical products and are available locally.

The wheels can be cleaned with hot water and a mild cleaning agent. After cleaning, the wheel bearings should be regularly lubricated with oil, as well as when the wheels begin to squeak.

Despite the solid construction and the use of resistant materials, the product is subject to wear. It is therefore recommended to have the product checked at regular intervals by a professional service. We recommend the following:

- Cleaning and disinfection of the product
- Checking the frame for bends, damage, wear or corrosion
- Checking wear parts and replacing them if necessary
- General retightening of components

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place at temperatures above 0 °C.

Transport

It is recommended to fold the product together for transport. The product should be transported in a stabilised and secured manner so that no sudden or dangerous movements can occur during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the product must be technically inspected and disinfected in accordance with Section 5.

6. Most Common Problems and Solutions

Problem	Possible Causes	Solution
The rollator is difficult to steer or push	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
The brakes do not work or only work on one side	The brake pads are worn, or the brakes are not properly adjusted	Adjust the brakes. Replace the brake pads if necessary (→ service centre).
The seat does not fit into the frame when you unfold the rollator	The lock is in an incorrect position or is defective	Check whether the lock is correctly adjusted. Replace it if necessary (→ service centre).

7. Disposal of the Product

The product cannot be disposed of with household waste and must be taken to the local recycling centre.

8. Warranty Conditions

Mobilex A/S offers a 2-year warranty for damage caused by production or material defects. The warranty does not apply in the case of improper repair or use of the product. Parts that are subject to normal wear are not covered by the warranty unless the wear is due to a manufacturing defect. If product modifications are made without our written consent, the CE marking and product warranty are no longer valid. In the event of product damage covered by this warranty, please contact your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injury or non-use of the product during repair. The warranty does not cover damage caused by failure to observe the user manual.

9. CE Declaration of Conformity

The product has passed the conformity assessment in accordance with EU Regulation MDR 2017/745 on medical devices and is CE marked. A declaration of conformity has been issued for the product, which you can access at the following link or via the QR code on the product label:

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