Black Panther Rollator

Art. 318101+318102

USER MANUAL

CE

Mobilex A/S
Grønlandsvej 5, 8660 Skanderborg,
Denmark,
www.mobilex.dk

Mobilex Sp. z o.o.
ul. Duńska 3/5, 91-204 Łódź
tel.: (0-42) 214-29-11
www.mobilex.pl

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Black Panther Rollator - Art. No. 318101, 318102

1. Introduction

You have chosen a high-quality Mobilex product and we are particularly pleased about that. This user manual contains a description and important information about the use of the medical device. It is intended to help you use the product correctly and safely. It is very important that you read the manual carefully before using the product. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly striving to improve and update its products. Therefore, we reserve the right to change specifications and technical data of the product without further notice.

Intended Use

This product is a medical device designed to increase the mobility of persons who have difficulty standing or walking. The rollator enables these persons to regain the ability to perform daily activities. The rollator is suitable for persons who require a stable walking aid due to various illnesses. The rollator should only be used as a support when standing and walking. The rollator seat can be used for sitting and resting. Any other use is prohibited.

Indications

The rollator is a medical aid that helps people with functional limitations. The indications for use are diseases that lead to mobility limitations and disabilities that impede independent standing and walking. Therefore, additional stable support is required when performing these activities. The indications cannot be related to a specific clinical picture and depend on the individual assessment of the health status of the potential user by a physician or physiotherapist. However, the main target groups are elderly people, persons with paresis, rheumatic diseases and other persons with limited mobility of the lumbar spine, knees or hips. The use of the product for any purpose other than that described above is prohibited.

Contraindications

Rollators are not suitable for persons who cannot maintain their balance while standing. The product must not be used if the user has problems with perception. Contraindications cannot be related to a specific clinical picture and depend on the individual assessment of the health status of the potential user. The choice of the appropriate orthopaedic aid for a specific clinical picture must always be made with the help of a specialist physician or physiotherapist. In addition, please observe the safety instructions in Chapter 2

Areas of Use

The product is suitable for indoor and outdoor use and can be used by private individuals in the home environment, in home care, in health and care facilities and in hospitals.

Medical Incidents and Adverse Events

Should adverse events occur in connection with the product, please contact the dealer or the manufacturer:

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark

Telephone: +45 87 93 22 20 Fax: +45 87 93 17 77 Email: info@mobilex.dk

2. Safety Requirements

Ensure that this user manual has been read by all persons who use the product.

The manufacturer assumes no responsibility for damage and/or injuries caused by failure to observe the user manual.

- Use the product only in perfect condition.
- If defects or faults are detected, you must immediately take the product out of service and contact the dealer.
- Follow all recommendations and warnings from the manufacturer.
- Observe the information on the product label.
- Use the product only for the described purpose.
- Do not climb on the product.
- Do not make any structural changes to the product unless you have written permission from the manufacturer for such modifications.
- The product and bag must not be loaded with more weight than specified in the technical data.
- All wheels MUST ALWAYS be on the ground during use. This ensures that the rollator is properly balanced and reduces the risk of accidents.
- Be careful when assembling or adjusting parts of the device.
- Always check that the wheel brakes are functioning properly before use.
- Only use the rollator when it is fully unfolded and the locking device is activated.
- The seat may only be used when the parking brakes are activated.
- Both brakes should always be activated or locked at the same time.
- It is not permitted to carry a person or an object on the seat while the rollator is in motion.
- The parking brakes must always be engaged when the rollator is not in use.

Delivery

The product is delivered in a cardboard box. The contents of the package are:

- 1 rollator with folded-in wheels incl. mounted seat, back strap and shopping bag (Picture 01)
- 1 user manual

Preliminary Inspection

Check the contents of the box against the above list. If at this point you find that a part is missing or damaged, contact your supplier immediately.

3. Description

Product Description

The Black Panther Compact Rollator has 4 wheels and a powder-coated aluminium frame in black. Due to the materials used, the entire rollator is corrosion-resistant. Thanks to the lightweight frame, it can be easily folded and offers simple storage and easy transport. The rollator runs smoothly and is sufficiently stable. The push handles can be adjusted in height. The rollator is supplied with a mounted seat, a back strap and a shopping bag. The rollator is equipped with reflectors on the push handles, the bag and the front wheel forks.

Special Feature - Wheel Folding Mechanism: The Black Panther Compact Rollator features an innovative wheel folding mechanism that allows all four wheels to be folded up against the frame. This significantly reduces the packed size of the rollator, making transport and storage particularly compact and convenient. This function distinguishes the Black Panther Compact from conventional rollators, which can only be folded sideways.

The rollator consists of the following elements (Image 02):

1. Push handle (Material: PP+TPR)
2. Brake lever (Material: PP)
3. Upper cane holder part
4. Seat tube
5. Button for handle height adjustment
6. Lower cane holder part
7. Brake
8. Rear wheel
9. Tilt pedal
10. Back strap (Material: PA)
11. Folding lock clamp
12. Seat with strap for folding (Material: PA)
13. Shopping bag (Material: PA)
14. Swivel front wheel
15. Reflectors
16. Wheel folding mechanism (locking lever on each wheel)
17. Assembly Description - Setup from Shipping Box

The Black Panther Compact Rollator is delivered in the box with folded-in wheels. Please carry out the following steps to make the rollator ready for use (Image 07 to 03):

Step 1 - Remove the rollator from the box: Carefully remove the rollator from the box and remove all packaging material. Place the rollator on a flat, stable surface.

Step 2 - Unfold the front wheels: Grasp the front wheel and the associated locking lever. Fold the wheel down into the operating position until the locking mechanism audibly clicks into place. Repeat the process for the second front wheel.

Step 3 - Unfold the rear wheels: Grasp the rear wheel and the associated locking lever. Fold the wheel down into the operating position until the locking mechanism audibly clicks into place. Repeat the process for the second rear wheel.

Step 4 - Unfold the frame: Press the folding lock button and push the seat tubes down until the frame unfolds and audibly "clicks", indicating that the rollator is locked in the unfolded position.

Step 5 - Inspection: Check that all four wheels are firmly locked in place and that the frame is fully unfolded and locked. All locks must be firmly engaged before the rollator can be used.

WARNING! All four wheels must be fully unfolded and firmly locked in place before the rollator is used. The seat tubes must lock firmly into place on both sides when unfolding the rollator. All parts must be securely locked before the rollator can be used. Failure to observe these warnings may result in injury.

Description of Function

Folding/Unfolding the Frame

To unfold the frame, press the folding lock button and push the seat tubes down until the frame unfolds and "clicks", indicating that the rollator is locked in the unfolded position. To fold the frame, pull the folding strap upwards and fold the frame together by pushing the seat tubes together until the folding lock button "clicks".

WARNING! The seat tubes must lock firmly into place on both sides when unfolding the rollator. All parts must be securely locked before the rollator can be used. Failure to observe these warnings may result in injury.

Folding in the Wheels for Compact Transport (Image 03 to 07)

The Black Panther Compact Rollator features an innovative wheel folding mechanism. To fold in the wheels, proceed as follows:

- First fold the frame together (see above).
- Release the locking lever on the first wheel.
- Fold the wheel upwards against the frame.
- Repeat the process for all four wheels.

To unfold the wheels again, fold each wheel downwards until the locking mechanism audibly clicks into place. Before use, check that all four wheels are securely locked.

WARNING! Always ensure that all four wheels are fully unfolded and firmly locked in place before using the rollator. Failure to do so may result in injury.

Push Handle Height Adjustment

To adjust the height of the armrests, press the height adjustment pin and move the armrest up or down by one step. Repeat the process until the desired setting is reached, and repeat the process for the second handle.

NOTE: Both handles should always be at the same height. The correct height of the push handles should always be at wrist height of the user when standing with arms at the side of the body.

Brakes

The brakes can be activated gradually by pulling the brake levers upwards with increasing force. The brakes are fully activated when the brake levers touch the handles and the rollator has come to a complete stop. To end the braking process, release the brake levers.

The rollator is also equipped with a parking brake function. To activate the parking brakes, push the brake lever downwards until it locks in this position. To release the parking brakes, pull the brake levers back to the neutral position.

WARNING! Always activate the parking brake when you are not using the rollator! If you park the rollator on an uneven surface, you must always activate the parking brake!

WARNING! The brakes must always be locked when the rollator is parked! Never use the seat when the brakes are not locked! Never lean back when sitting. It is recommended to sit only on the front part of the seat. Do not sit on the bag.

Tilt Pedal

The tilt pedal makes it easier for the user to overcome small obstacles by tilting the rollator backwards. The tilt pedal is integrated into the frame.

Reflectors

The rollator is equipped with reflectors on all sides, including on the push handles, the bag and the front wheel forks.

Accessories

There are no accessories for this device.

4. Technical Data

Black Panther Compact Rollator	Art. 318101	Art. 318102
<i>Push handles height adjustable / Height of product</i>	81 - 96.5 cm	84 - 99.5 cm
<i>Recommended for user height from</i>	140 - 187 cm	150 - 195 cm
<i>Seat height</i>	54.5 cm	61.5 cm
<i>Total width open / folded</i>	63 / 26 cm	63 / 26 cm
<i>Inner width between push handles</i>	47.5 cm	47.5 cm
<i>Length (depth)</i>	68 cm	68 cm
<i>Weight (+ bag 380 g)</i>	8.4 kg	8.5 kg
<i>Max. load capacity (rollator / bag)</i>	150 kg / 5 kg	150 kg / 5 kg
<i>Brakes</i>	2 lockable with parking function	2 lockable with parking function
<i>Wheel sizes (front and rear)</i>	200 x 35 mm	200 x 35 mm
<i>Seat width</i>	45 cm	45 cm
<i>Seat depth</i>	25 cm	25 cm
<i>Turning circle</i>	84 cm	84 cm
<i>Tube diameter</i>	Cross for frame Ø22 mm / Frame Ø26.25x52 mm	Cross for frame Ø22 mm / Frame Ø26.25x52 mm
<i>Material</i>	Aluminium frame, wheels made of PP and rubber	Aluminium frame, wheels made of PP and rubber

Product Label

Example of the product label (see Image 08)

Explanation of symbols (see Image 09)

Symbol	Description	Meaning
1.	QR code	Link to homepage
2.	Manufacturer symbol	Manufacturer
3.	i-symbol	See user manual
4	CE	CE product
5	Basket with 2 kg symbol	Max. load basket
6.	House/outdoor symbol	For indoor/outdoor use
7.	Date symbol	Production date
8.	Kg. with down arrow	Max. Load
9.	Arrow symbol	Max. width
10.	SN	Serial number
11.	REF	Article number
12.	MD	Medical device

5. Maintenance and Cleaning

Normal dirt can be removed with commercially available cleaning agents and a sponge or soft cloth. Check the specific product information and use only commercially available cleaning products that are suitable for cleaning. For disinfection, use only disinfectants that are certified for use with medical products and are available locally.

The wheels can be cleaned with hot water and a mild cleaning agent. After cleaning, the wheel bearings should be regularly lubricated with oil, as well as when the wheels begin to squeak.

The wheel folding mechanism should be regularly checked for proper function. Ensure that the locking levers are free from dirt and deposits and that the mechanism operates smoothly. If necessary, the moving parts of the folding mechanism can be treated with a light lubricant.

Despite the solid construction and the use of resistant materials, the product is subject to wear. It is therefore recommended to have the product checked at regular intervals by a professional service. We recommend the following:

Cleaning and disinfection of the product

- Checking the frame for bends, damage, wear or corrosion
- Checking wear parts and replacing them if necessary
- General retightening of components
- Checking the wheel folding mechanism for proper function and secure fit

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place at temperatures above 0 °C.

Transport

It is recommended to fold the product together and fold in the wheels for transport in order to achieve the most compact packed size possible. The product should be transported in a stabilised and secured manner so that no sudden or dangerous movements can occur during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the product must be technically inspected and disinfected in accordance with Section 5.

6. Most Common Problems and Solutions

Problem	Possible Cause	Solution
<i>The rollator is difficult to steer or push</i>	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
<i>The brakes do not work or only work on one side</i>	The brake pads are worn, or the brakes are not properly adjusted	Adjust the brakes. Replace the brake pads if necessary (→ service centre).
<i>The seat does not fit into the frame when you unfold the rollator</i>	The lock is in an incorrect position or is defective	Check whether the lock is correctly adjusted. Replace it if necessary (→ service centre).
<i>A wheel does not lock firmly when unfolding</i>	The locking mechanism is dirty or damaged	Clean the locking mechanism. Check for damage. Contact the service centre if necessary.
<i>The wheel folding mechanism is stiff</i>	Contamination or lack of lubrication of the mechanism	Clean the mechanism and apply a light lubricant to the moving parts.

7. Disposal of the Product

The product cannot be disposed of with household waste and must be taken to the local recycling centre.

8. Warranty Conditions

Mobilex A/S offers a 2-year warranty for damage caused by production or material defects. The warranty does not apply in the case of improper repair or use of the product. Parts that are subject to normal wear are not covered by the warranty unless the wear is due to a manufacturing defect. If product modifications are made without our written consent, the CE marking and product warranty are no longer valid. In the event of product damage covered by this warranty, please contact your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injury or non-use of the product during repair. The warranty does not cover damage caused by failure to observe the user manual.

9. CE Declaration of Conformity

The product has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 on medical devices and is CE marked. A declaration of conformity has been issued for the product, which you can access via the QR code on the product label or at https://mobilex.dk/medias/2026-02/318101_02_ce_20010.pdf

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