USER MANUAL

ST RAMPS: ST-091, ST-122, ST-152


DE - EN - DA - SV - NO - NL - FR - IT - ES - PT - PL

CE

Reference images

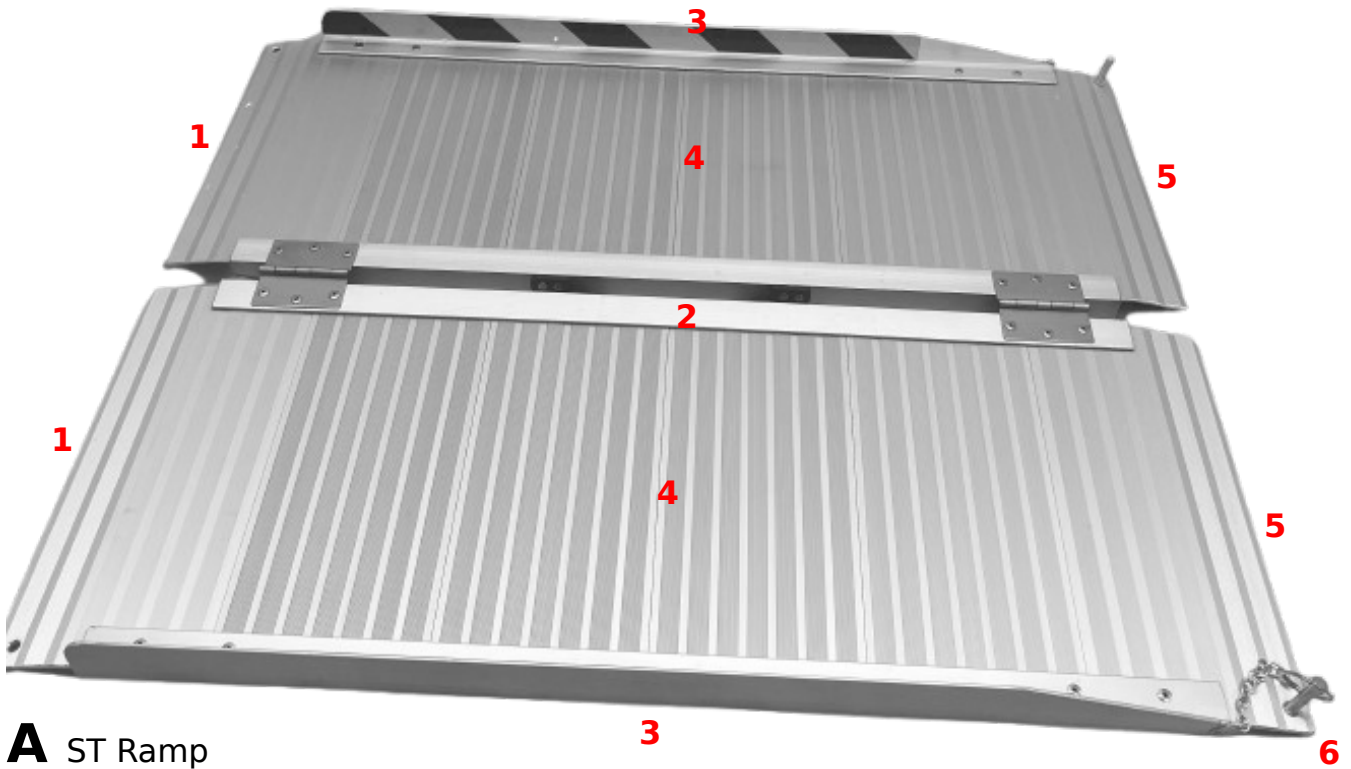A ST Ramp

B Anti-slip

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1. INTRODUCTION

You have chosen a high-quality Mobilex product and we are particularly pleased about this. This user manual contains a description and important information about the use of the medical device. It is intended to help you use the product correctly and safely. It is very important that you read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly striving to improve and update its products. Therefore, we reserve the right to change specifications and technical data of the product without further notice.

Intended Use

To assist patients in wheelchairs or scooters in overcoming elevations.

Indications / Contraindications

Ramps are suitable for people with mobility impairments who are forced to use mobility aids such as wheelchairs, scooters, rollators, etc. due to problems with walking, standing or sitting. Ramps are suitable for eliminating architectural barriers by overcoming height differences that a wheelchair user must overcome, and enable the wheelchair user to reach places that would normally be inaccessible to them. The ramp is intended for independent use or use with the help of an accompanying person. It is prohibited to use the ramp for purposes other than those stated above.

Contraindications cannot be linked to a specific clinical picture. The choice of a suitable ramp type and length must always be made with the help of a specialist doctor or physiotherapist. Also note the safety instructions in Chapter 2.

Areas of Application

The product can be used in home care, health and care facilities, and hospitals.

Medical Incidents and Adverse Events

Should adverse events occur in connection with the product, please contact the dealer or the manufacturer:

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark Phone: +45 87 93 22 20 Fax: +45 87 93 17 77 Email: info@mobilex.dk

Serious incidents must also be reported to the competent authority of the Member State in which the user and/or patient is resident in accordance with MDR 2017/745.

2. SAFETY INSTRUCTIONS

WARNING - Please observe the following safety instructions:

- Ensure that all persons using the product have read these instructions.
- The manufacturer assumes no responsibility for damage and/or injuries resulting from non-compliance with the operating instructions.
- Only use the product in perfect condition.
- If you notice any defects or faults, you must contact the dealer immediately.
- Follow all recommendations and warnings from the manufacturer.
- Observe the information on the product label.
- Only use the product for the described purpose.
- Do not make any structural changes to the product unless you have written permission from the manufacturer for such modifications.
- With the various settings that are possible on the product, the user must ensure that they do not trap their fingers or other body parts between two moving elements of the product.
- The product may only be used on a stable surface.
- The ramps must not be loaded with more than the "maximum load" specified in the technical data and on the label.
- The ramps can be used to overcome the following maximum heights:
ST-091: 18 cm ST-122: 24 cm ST-152: 30 cm
- Be careful when folding and unfolding the ramp. There is a risk that you may trap your fingers or

- cut yourself between parts of the ramp.
- Check the stability and correct positioning of the ramp before each use.
- Do not use the ramp in adverse weather conditions (ice, heavy rain) that could make the surface slippery.

Delivery

Each product package contains: 1 ramp, 1 user manual.

Preliminary Check

Check the contents of the box against the above list. If you find at this point that a part is missing or damaged, contact your supplier immediately.

3. DESCRIPTION

Product Description

ST-Ramps are foldable platforms made of aluminum. They have a non-slip, ribbed surface.

The ramp consists of:

1. Upper support
2. Carrying handle (under the ramp)
3. Side protection
4. Non-slip, ribbed surface
5. Lower support
6. Locking mechanism (prevents the ramp from sliding away)

Function Description

Unfolding and Placing the Ramp

Place the ramp down and unfold it so that the carrying handle is on the underside.

To fold the ramp again, simply pull one side over the other so that the carrying handle is accessible.

Positioning the Ramp

Place the upper support on the elevated level and the lower support on the ground level. Make sure the ramp is stable and standing on a stable surface before using it.

Assembly

This product is delivered already assembled and therefore does not require assembly instructions.

Accessories

No accessories are available for the ST ramps.

4. TECHNICAL DATA

ST-Ramps Specifications:

Specification	ST-091	ST-122	ST-152
<i>Length</i>	91.4 cm	121.9 cm	152.4 cm
<i>Width</i>	73.8 cm	73.8 cm	73.8 cm
<i>Surface Width</i>	73 cm	73 cm	73 cm
<i>Height of Side Protection</i>	3.5 cm	3.5 cm	3.5 cm
<i>Max. Elevation / Height</i>	12° / 18 cm	12° / 24 cm	12° / 30 cm
<i>Weight</i>	8 kg	10.4 kg	13.1 kg
<i>Max. Load</i>	272 kg	272 kg	272 kg
<i>Folded Size (LxWxH)</i>	101 x 43 x 10.5 cm	131 x 43 x 11 cm	162.5 x 43 x 11 cm

5. MAINTENANCE & CARE

Normal dirt can be removed with commercially available cleaning agents and a sponge or soft cloth. Check the specific product information and only use commercially available cleaning agents suitable for cleaning and disinfection (do not use alcohol-containing products or solvents or abrasives).

The product basically requires no maintenance. Despite the solid construction and the use of resistant materials, the product is subject to wear. It is therefore recommended to have the product checked by a professional service at regular intervals.

We recommend checking the following:

- Cleaning and disinfection of the product
- The frame/product should be checked for bending, damage, wear or corrosion
- Wear parts are checked and replaced if necessary
- Checking the folding elements and carrying handles for proper function
- Inspection of the non-slip surface for damage or wear

Expected Service Life: Normally up to 10 years, depending on use.

Storage: We recommend storage in a dry place at a temperature above 0 degrees and protected from intense sunlight.

Transport

The ramp can be locked in the folded position. It should be transported stabilized and secured so that sudden or dangerous movements cannot occur during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, it must be checked according to point 5.

6. MOST COMMON PROBLEMS AND SOLUTIONS

Problem	Possible Causes	Solutions
<i>Ramp is not stable</i>	The surface under the ramp is not level	Check the position of the ramp. Place it on a flat, smooth surface.
<i>Ramp is not stable</i>	The ramp is being used upside down	Check whether the ramp is positioned correctly
<i>Ramp will not open</i>	The fastening strap is not released	Release the fastening strap
<i>Ramp will not fold</i>	Obstacles between the folding parts	Remove dirt or foreign objects from the folding areas
<i>Surface is slippery</i>	Moisture, ice or contamination	Clean and dry the surface before use

7. DISPOSAL OF THE PRODUCT

The product must not be disposed of with household waste, but must be taken to the local recycling center. The ramp consists mainly of aluminum and can be recycled. Observe the applicable local regulations for the disposal of aluminum products.

8. WARRANTY CONDITIONS

Mobilex A/S offers a 2-year warranty for damage caused by production or material defects. The warranty is not valid for improper repair or use of the product. Parts subject to normal wear are not covered by the warranty unless the wear is due to a manufacturing defect. If product changes are made without our written consent, the CE marking and product warranty become invalid. In case of product damage covered by this warranty, please contact your dealer or Mobilex A/S directly. The warranty does not cover transportation costs and does not include compensation for personal injury or non-use of the product during repair. The warranty does not cover damage resulting from non-compliance with the operating instructions.

Contact:

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark Phone: +45 87 93 22 20 Fax: +45 87 93 17 77 Email: info@mobilex.dk Web: www.mobilex.dk

9. CE DECLARATION

The product has passed the conformity assessment according to the EU regulation: MDR 2017/745 on medical devices and is CE marked. A declaration of conformity has been issued for the product, which you can access at:

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